

SCOTTSDALE POLICE DEPARTMENT CRIME LABORATORY BLOOD ALCOHOL FACE SHEET

ANALYSIS DATE 10/31/2023 SEQUENCE NAME 31Oct23

EQUIPMENT

Pipettor Hamilton ML600EH7497 Hamilton ML600GJ10749
 Gas Chromatograph Agilent US14173023

INSTRUMENT CALIBRATION

Vial 1 0.02 calibrator Lot FN03122113 Coefficient of determination (r²)
 Vial 2 0.10 calibrator Lot FN11172002 0.99942
 Vial 3 0.20 calibrator Lot FN03132302
 Vial 4 0.40 calibrator Lot FN03052102

CALIBRATION VERIFICATION AND RESOLUTION TEST

Vial	Sample	Expected result	Measured result	Manufacturer/lot
5	Blank	Not detected	Not detected	SPD lab 011823WB
6	Volatiles mixture	6 compounds	6 compounds	SPD lab 050721MIX
7	Aqueous control	0.400 g/dL	0.396 g/dL	Lipomed 30012020-A
8	Aqueous control	0.040 g/dL	0.039 g/dL	Lipomed 14082019-B
9	Blood control	0.199 g/dL	0.202 g/dL	ACQ 4110320133/11
20	Aqueous control	0.080 g/dL	0.078 g/dL	Lipomed 20012020-B
31	Aqueous control	0.080 g/dL	0.078 g/dL	Lipomed 20012020-B
42	Aqueous control	0.400 g/dL	0.399 g/dL	Lipomed 30012020-A
43	Aqueous control	0.040 g/dL	0.039 g/dL	Lipomed 14082019-B
44	Blood control	0.199 g/dL	0.201 g/dL	ACQ 4110320133/11
45	Blank	Not detected	Not detected	SPD lab 092223AQ

SUBJECT SAMPLES

Subjects in the sequence 15 Subjects requiring reanalysis 0

ADDITIONAL NOTES: All testing proceeded as expected.

Run valid Run invalid [Signature] Analyst
 Run valid Run invalid [Signature] Technical Reviewer


Scottsdale Police Department Crime Laboratory Sequence Quality Assurance Summary

SEQUENCE NAME: 31Oct23

ANALYST: Abbott *AA*

Sample Name	Vial	Measured Value (g/dL)	Expected Value (g/dL)	Percent Difference	Absolute Difference (g/dL)
blank 092223AQ	5	negative	negative	-	-
0.400 30012020-A	7	0.396	0.400	1.00	-0.004
0.040 14082019-B	8	0.039	0.040	2.50	-0.001
0.199 ACQ 4110320133/11	9	0.202	0.199	1.51	0.003
0.080 20012020-B	20	0.078	0.080	2.50	-0.002
0.080 20012020-B	31	0.078	0.080	2.50	-0.002
0.400 30012020-A	42	0.399	0.400	0.25	-0.001
0.040 14082019-B	43	0.039	0.040	2.50	-0.001
0.199 ACQ 4110320133/11	44	0.201	0.199	1.01	0.002
blank 011823WB	45	negative	negative	-	-

Calibrator	Ethanol Area	N-propanol Area	Ratio
0.020	11.827	151.739	0.078
0.100	59.687	149.535	0.399
0.200	138.587	165.334	0.838
0.400	238.322	147.394	1.617

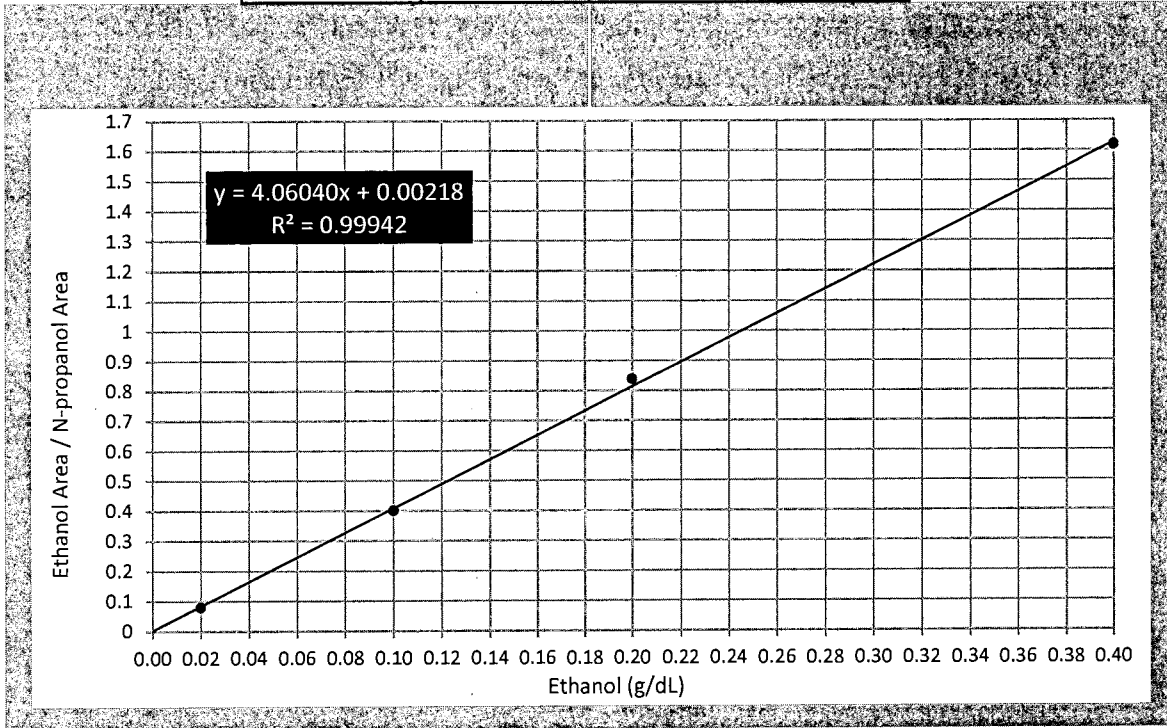

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: FN03122113	Vial: 1
Description: 0.020 calibrator	Sequence: 31Oct23
Method: ethanol quant.M	Injection date: 10/31/2023 4:08:33 PM
Instrument: US14173023 CN14160045	Analyst: Abbott <i>ds</i>
Data file: C:\Chem32\1\Data\31Oct23\1.D	

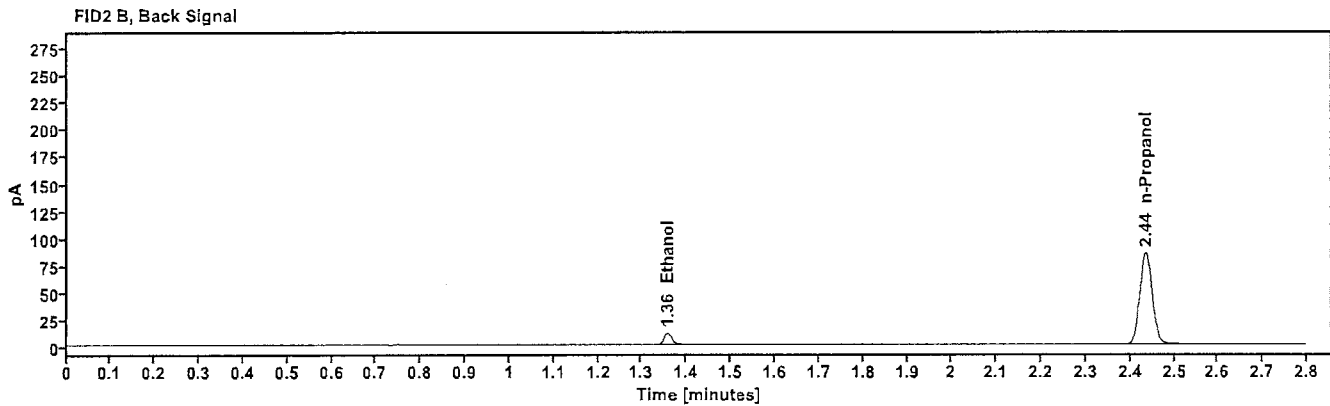
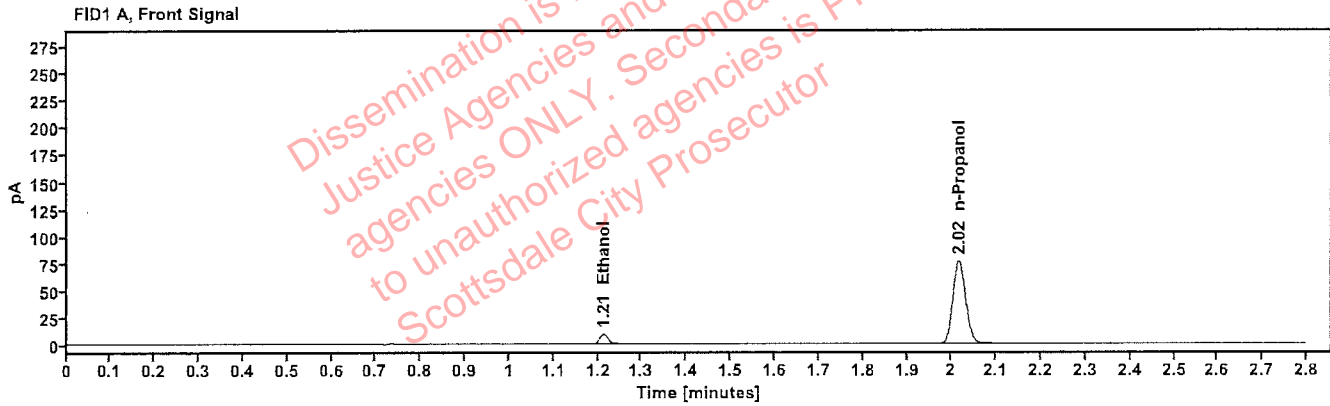


Table 1: FID 1 A (column DB-ALC1)

Compound	Time (min)	Peak Area
Ethanol	1.212	11.827
n-Propanol	2.017	151.739

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.358	12.964
n-Propanol	2.436	166.500

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: FN11172002	Vial: 2
Description: 0.100 calibrator	Sequence: 31Oct23
Method: ethanol quant.M	Injection date: 10/31/2023 4:12:33 PM
Instrument: US14173023 CN14160045	Analyst: Abbott <i>Abbott</i>
Data file: C:\Chem32\1\Data\31Oct23\2.D	

Dissemination is restricted to Criminal Justice Agencies and authorized agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

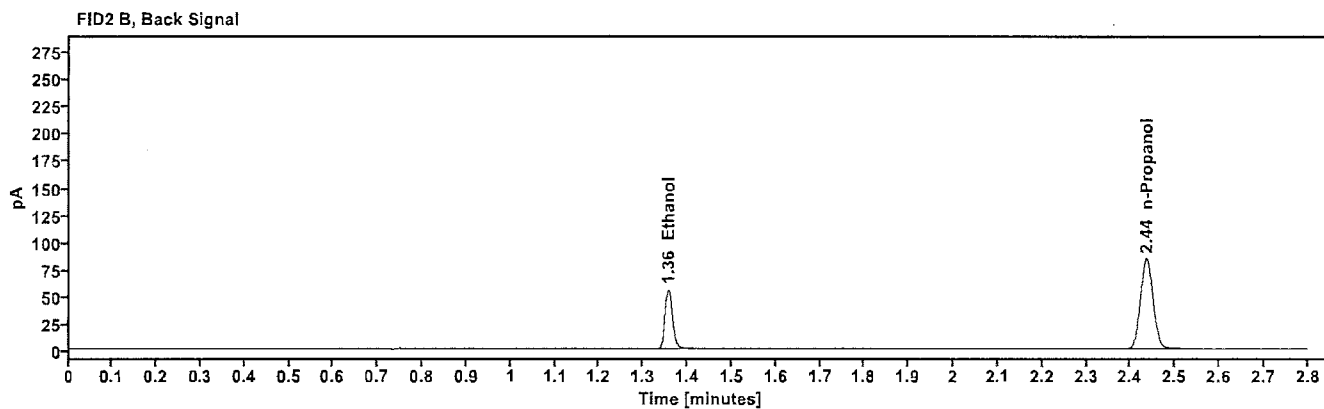
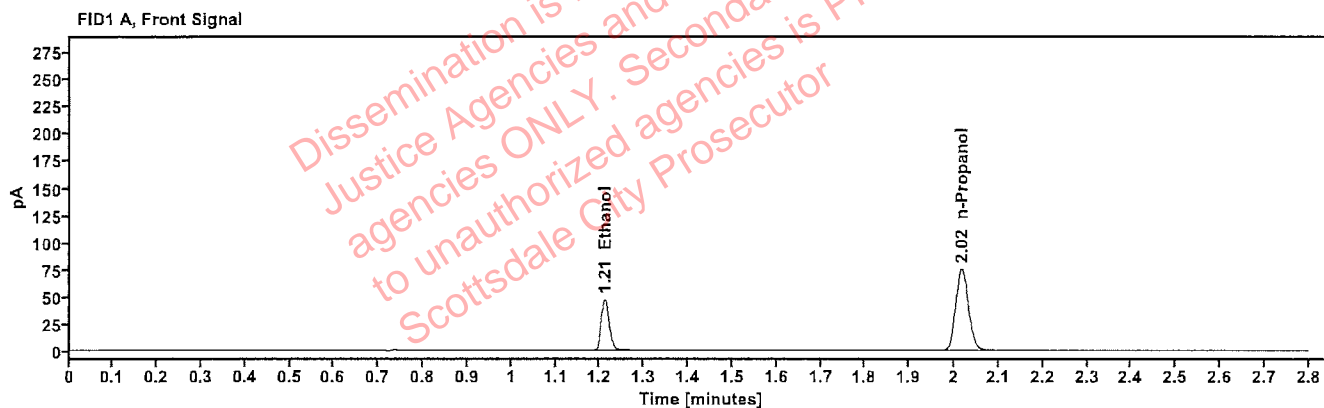


Table 1: FID 1 A (column DB-ALC1)

Compound	Time (min)	Peak Area
Ethanol	1.211	59.687
n-Propanol	2.017	149.535

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.356	65.715
n-Propanol	2.436	164.081

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: FN03132302	Vial: 3
Description: 0.200 calibrator	Sequence: 31Oct23
Method: ethanol quant.M	Injection date: 10/31/2023 4:16:33 PM
Instrument: US14173023 CN14160045	Analyst: Abbott <i>JA</i>
Data file: C:\Chem32\1\Data\31Oct23\3.D	

Dissemination is restricted to Criminal Justice Agencies and authorized agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

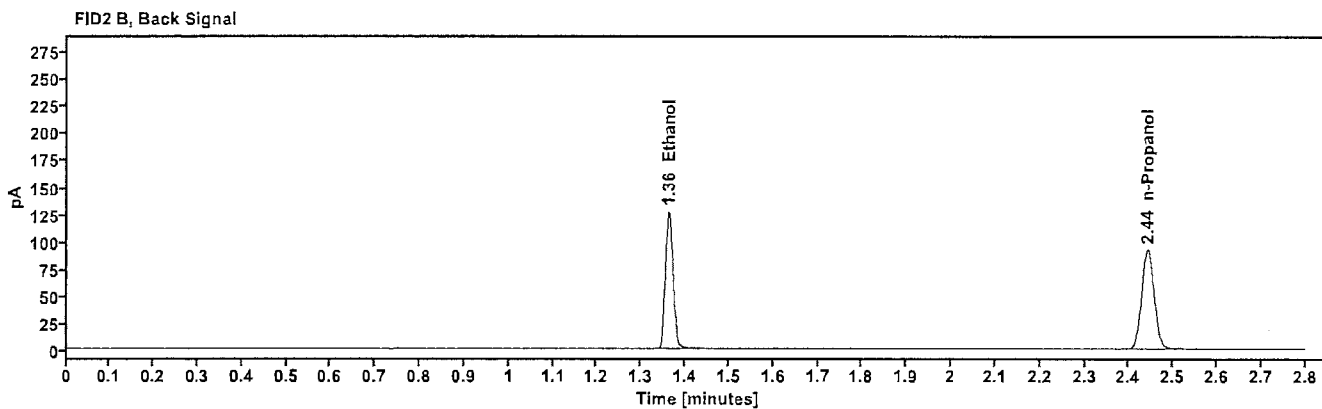
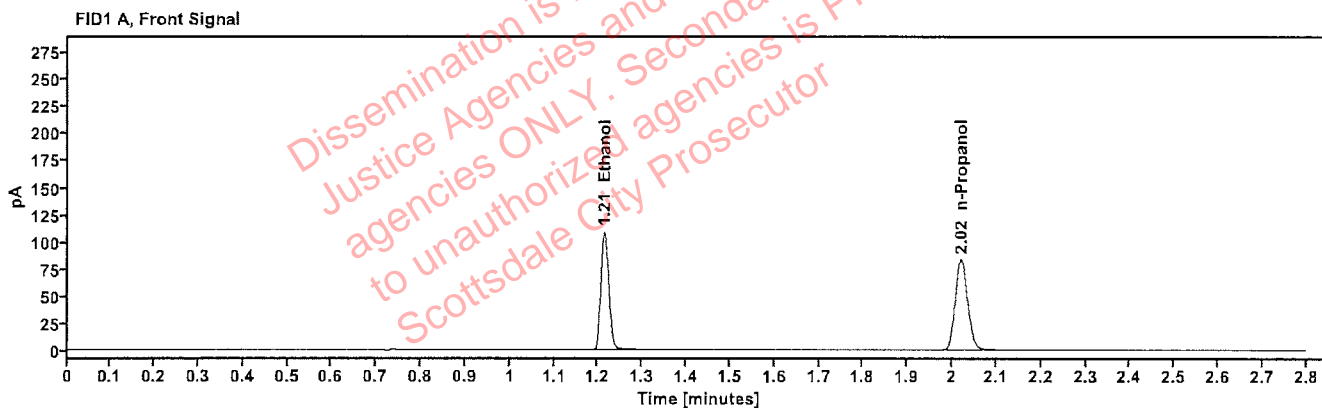


Table 1: FID 1 A (column DB-ALC1)

Compound	Time (min)	Peak Area
Ethanol	1.214	138.587
n-Propanol	2.021	165.334

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.363	152.749
n-Propanol	2.442	181.224

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: FN03052102	Vial: 4
Description: 0.400 calibrator	Sequence: 31Oct23
Method: ethanol quant.M	Injection date: 10/31/2023 4:20:34 PM
Instrument: US14173023 CN14160045	Analyst: Abbott <i>DA</i>
Data file: C:\Chem32\1\Data\31Oct23\4.D	

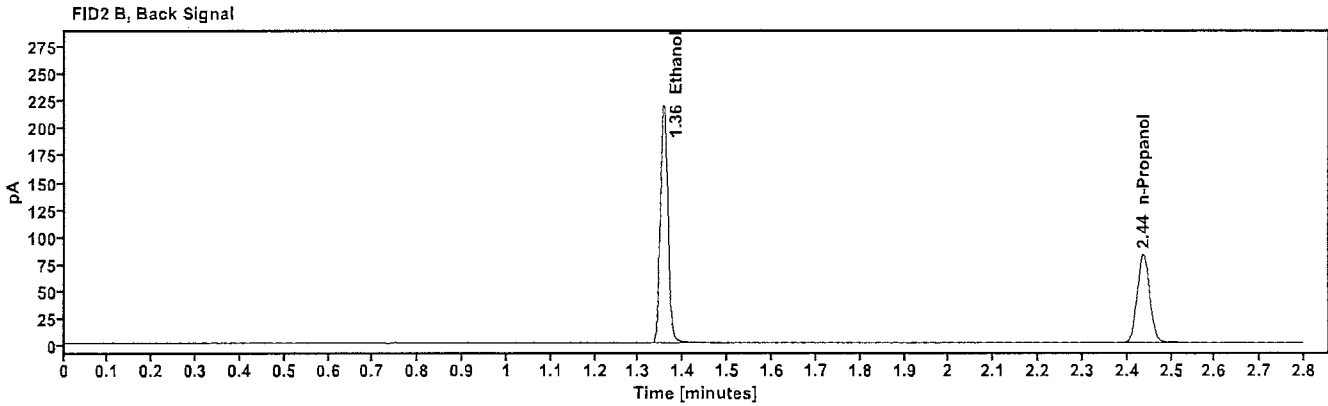
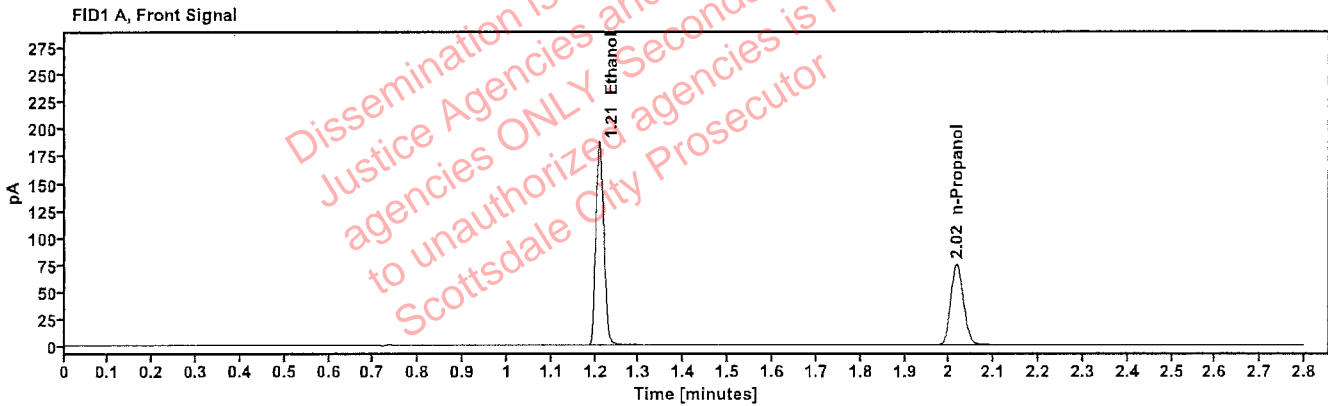


Table 1: FID 1 A (column DB-ALC1)

Compound	Time (min)	Peak Area
Ethanol	1.209	238.322
n-Propanol	2.017	147.394

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.356	263.071
n-Propanol	2.436	161.510

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 011823WB	Vial: 5
Description: Negative	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 4:24:34 PM
Data file: C:\Chem32\1\Data\31Oct23\5.D	Analyst: Abbott <i>AB</i>

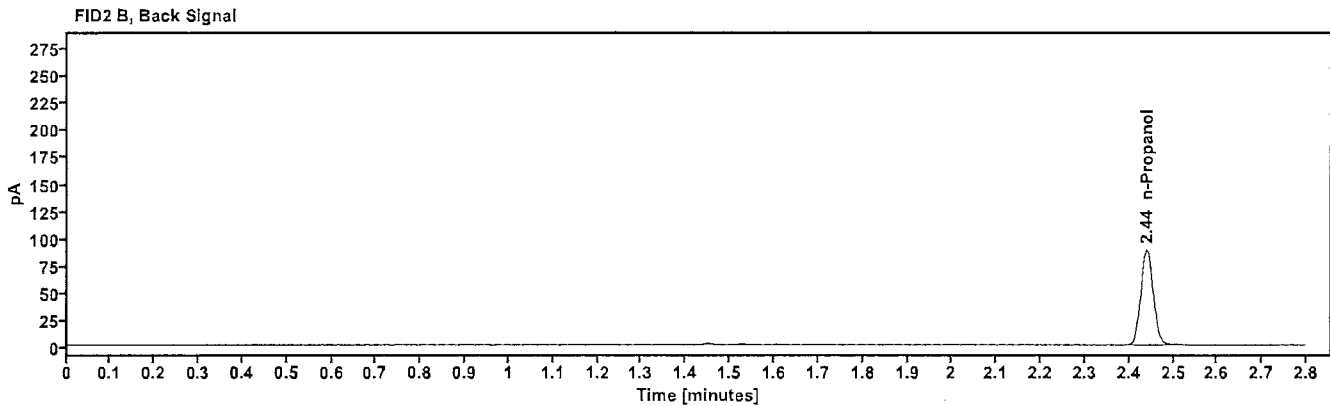
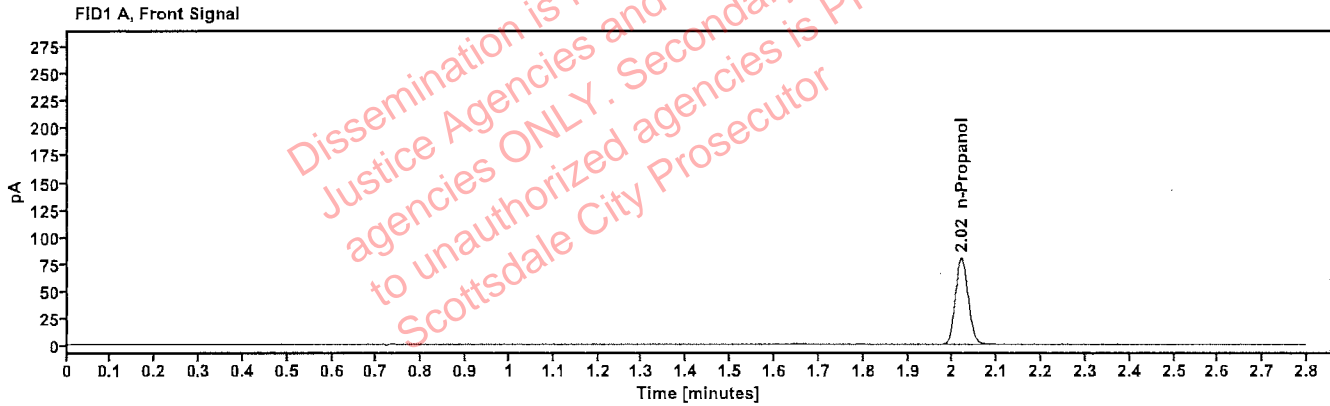


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
n-Propanol	—	2.020	158.485

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
n-Propanol	2.440	173.818

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 050721MIX	Vial: 6
Description: Volatiles mix	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 4:28:33 PM
Data file: C:\Chem32\1\Data\31Oct23\6.D	Analyst: Abbott

Abbott

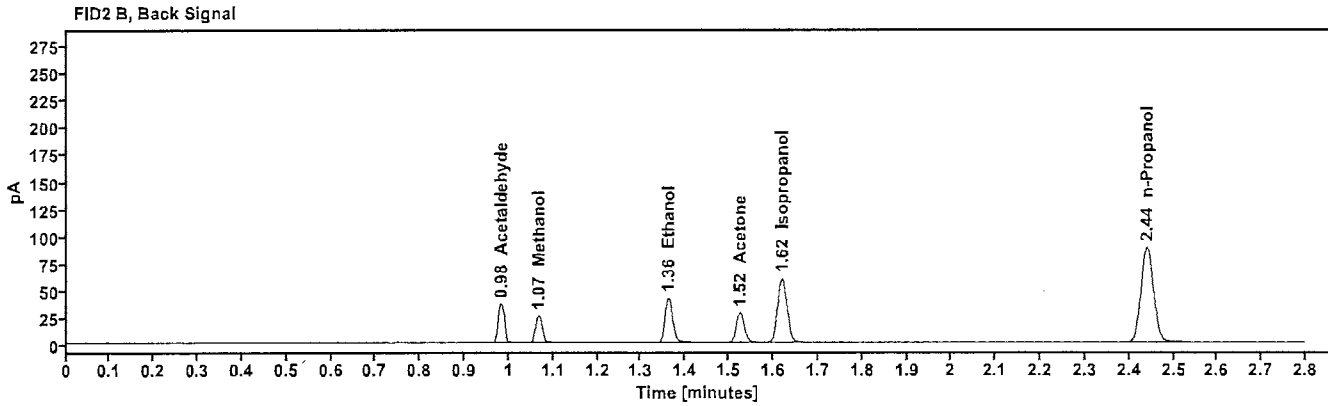
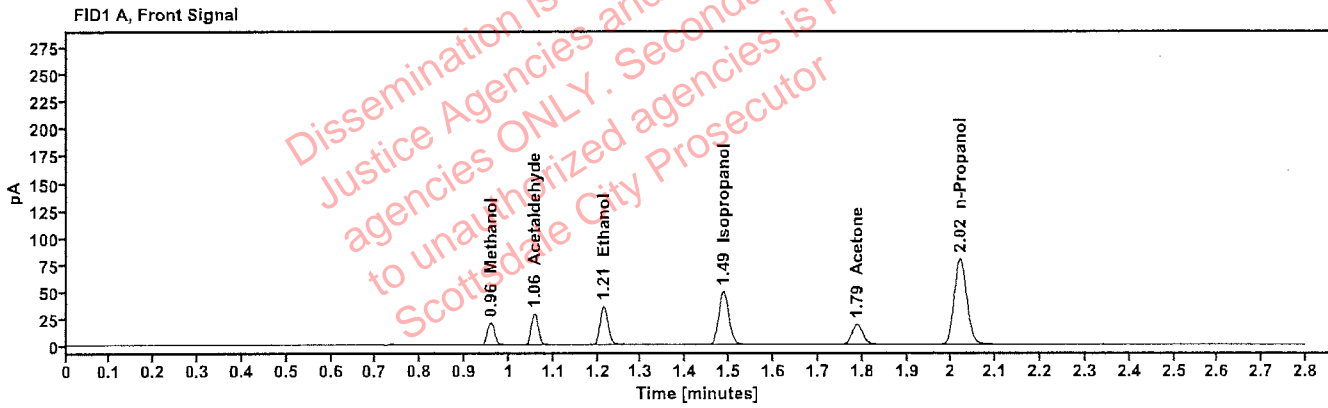


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
Methanol	-----	0.960	23.815
Acetaldehyde	-----	1.057	31.091
>Ethanol	0.0701	1.214	45.412
Isopropanol	-----	1.487	78.632
Acetone	-----	1.788	32.539
n-Propanol	-----	2.020	158.309

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Acetaldehyde	0.983	34.847
Methanol	1.067	26.470
Ethanol	1.361	50.021
Acetone	1.525	35.890
Isopropanol	1.620	87.260
n-Propanol	2.440	173.536

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 30012020-A	Vial: 7
Description: 0.400	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 4:32:49 PM
Data file: C:\Chem32\1\Data\31Oct23\7.D	Analyst: Abbott

Dissemination is restricted to Criminal Justice Agencies and authorized agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

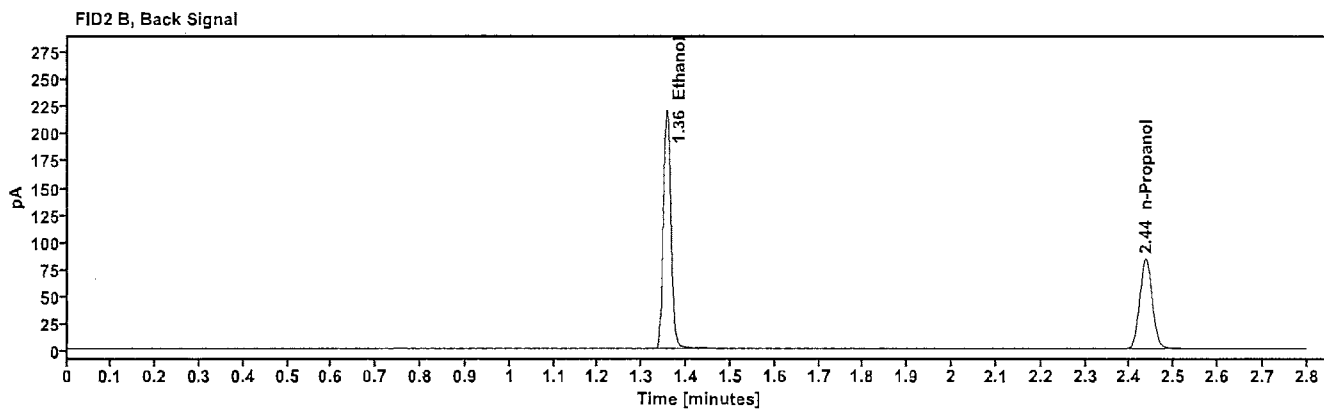
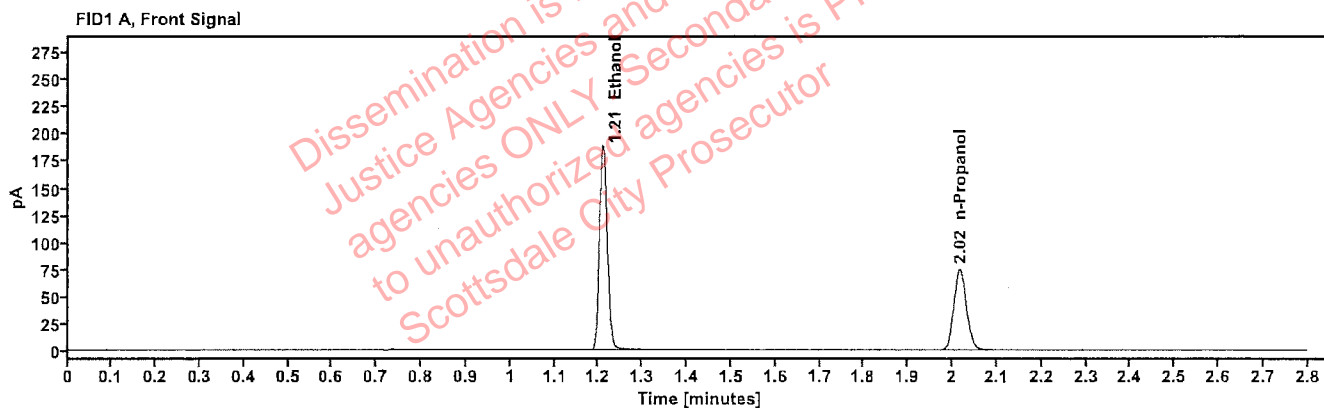


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.3962	1.209	238.823
n-Propanol	-----	2.017	148.252

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.356	263.527
n-Propanol	2.436	162.473

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 14082019-B	Vial: 8
Description: 0.040	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 4:36:47 PM
Data file: C:\Chem32\1\Data\31Oct23\8.D	Analyst: Abbott

OR

Dissemination is restricted to Criminal Justice Agencies and authorized agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

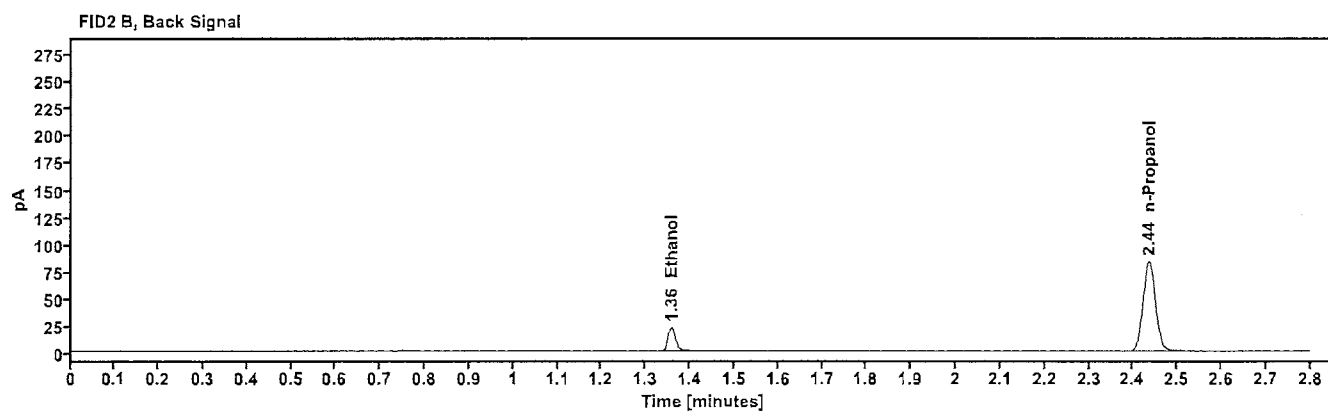
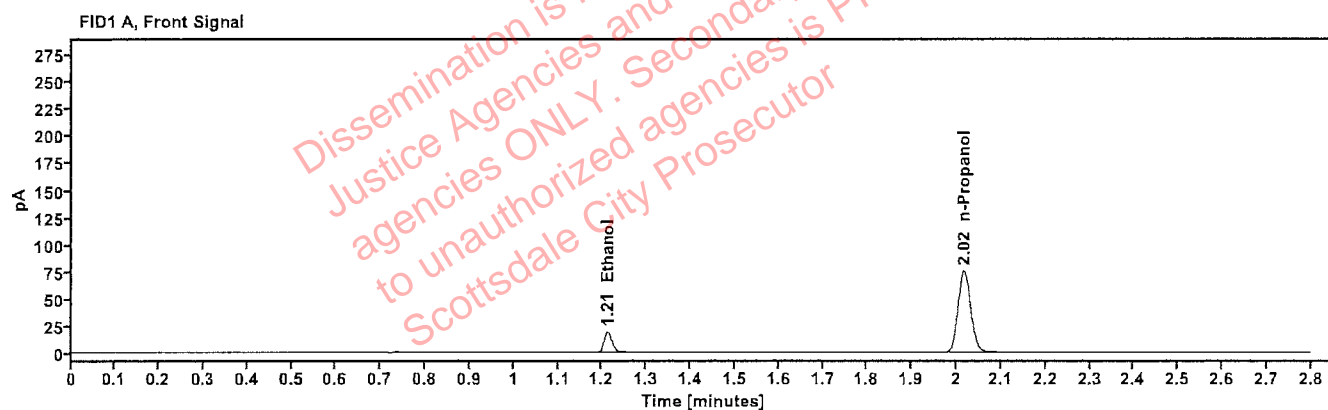


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0390	1.212	23.918
n-Propanol	-----	2.017	149.016

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.357	26.292
n-Propanol	2.436	163.443

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 4110320133/11	Vial: 9
Description: 0.199	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 4:40:47 PM
Data file: C:\Chem32\1\Data\31Oct23\9.D	Analyst: Abbott

Handwritten initials

Dissemination is restricted to Criminal Justice Agencies and authorized agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

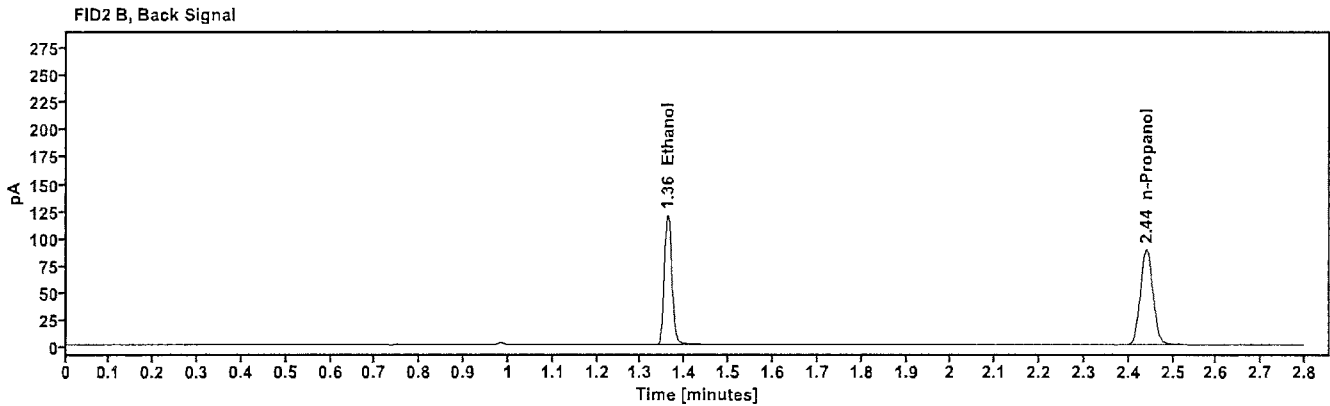
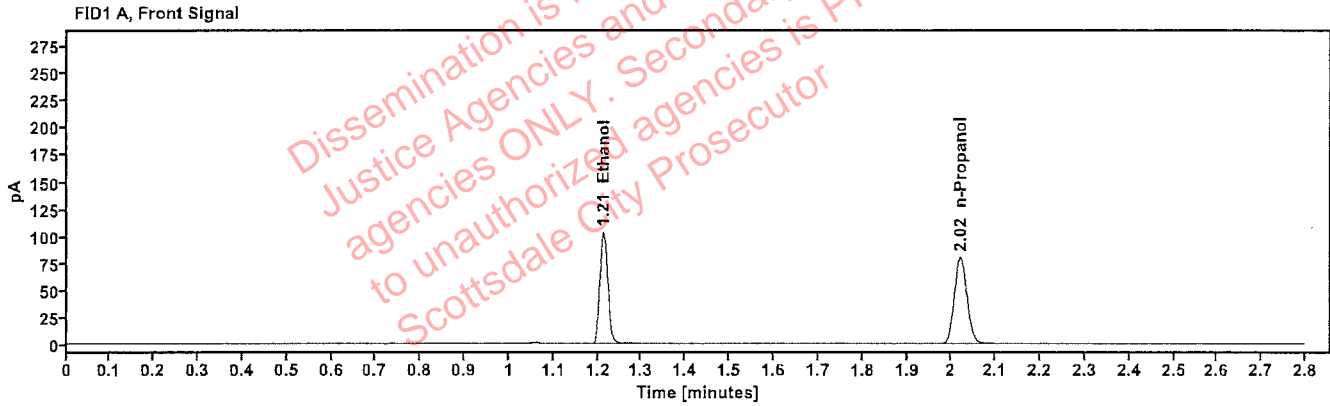


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2021	1.213	130.815
n-Propanol	-----	2.020	158.997

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	144.214
n-Propanol	2.440	174.399

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 20012020-B	Vial: 20
Description: 0.080	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 5:25:18 PM
Data file: C:\Chem32\1\Data\31Oct23\20.D	Analyst: Abbott

Abbott

Dissemination is restricted to Criminal Justice Agencies and authorized law enforcement agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

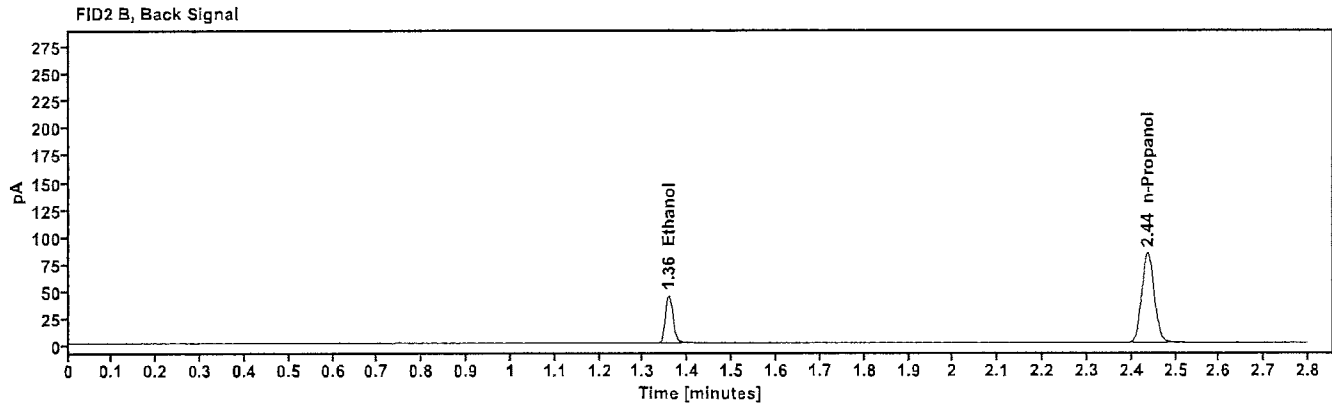
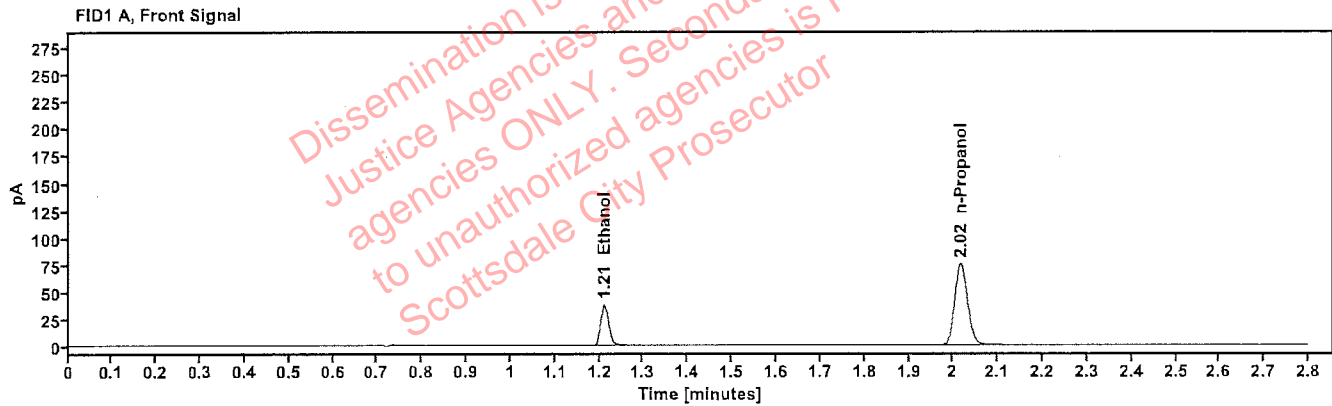


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0783	1.211	47.838
n-Propanol	----	2.017	149.513

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.357	52.650
n-Propanol	2.436	163.775

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 20012020-B	Vial: 31
Description: 0.080	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 6:09:48 PM
Data file: C:\Chem32\1\Data\31Oct23\31.D	Analyst: Abbott

Abbott

Dissemination is restricted to Criminal Justice Agencies and authorized agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

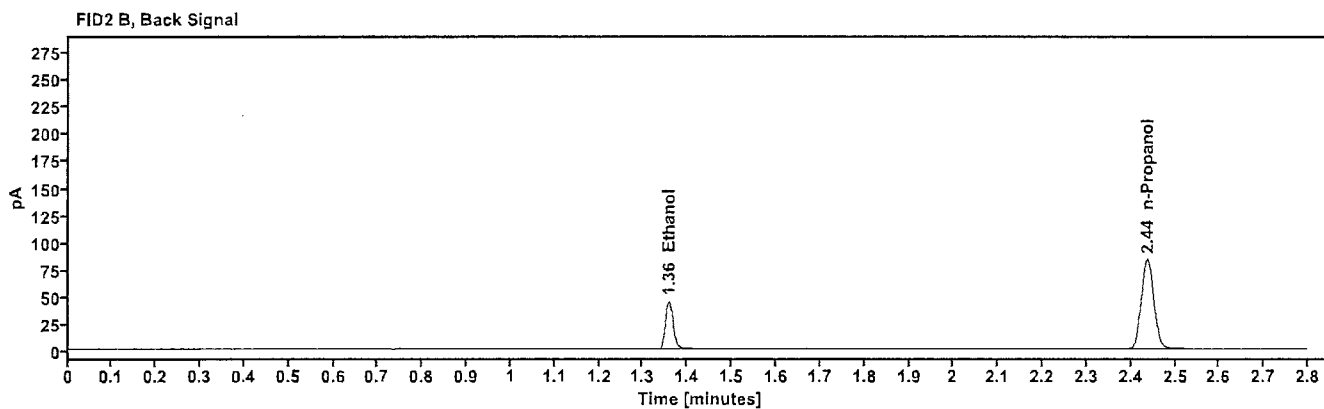
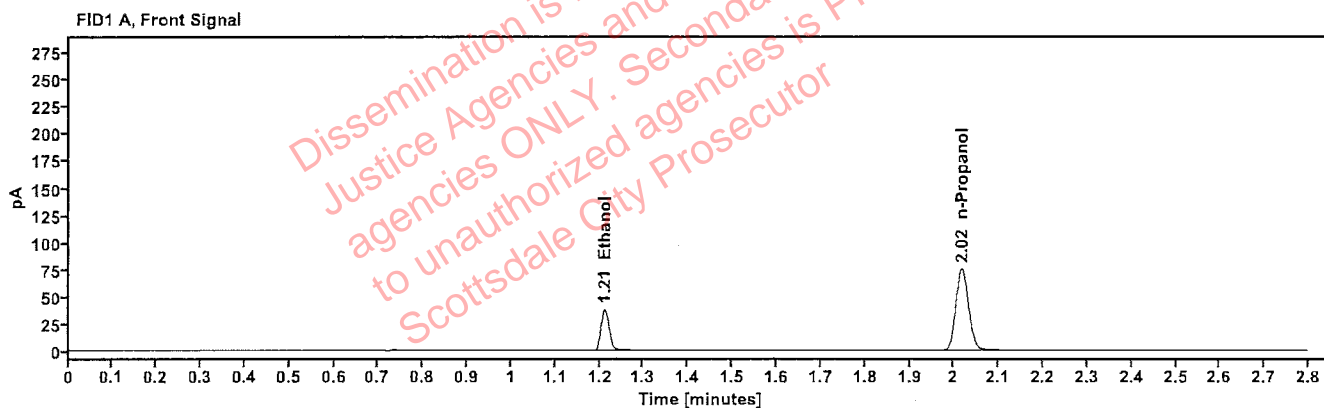


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0789	1.211	48.131
n-Propanol	-----	2.018	149.302

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.358	52.868
n-Propanol	2.437	163.560

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	30012020-A	Vial:	42
Description:	0.400	LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:54:03 PM
Data file:	C:\Chem32\1\Data\31Oct23\42.D	Analyst:	Abbott DA

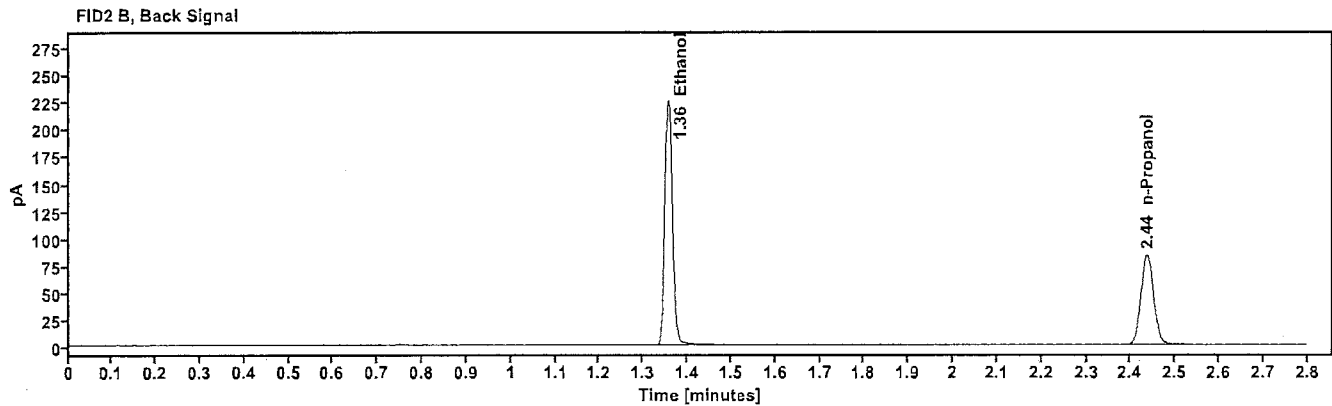
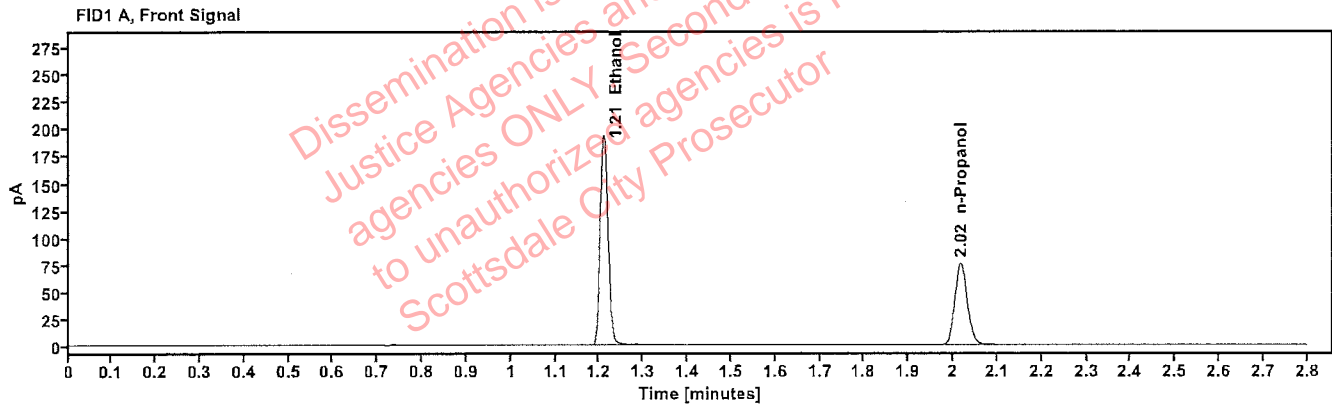


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.3993	1.210	243.564
n-Propanol	-----	2.017	150.040

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.357	269.136
n-Propanol	2.437	164.392

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 14082019-B	Vial: 43
Description: 0.040	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 6:58:18 PM
Data file: C:\Chem32\1\Data\31Oct23\43.D	Analyst: Abbott

AK

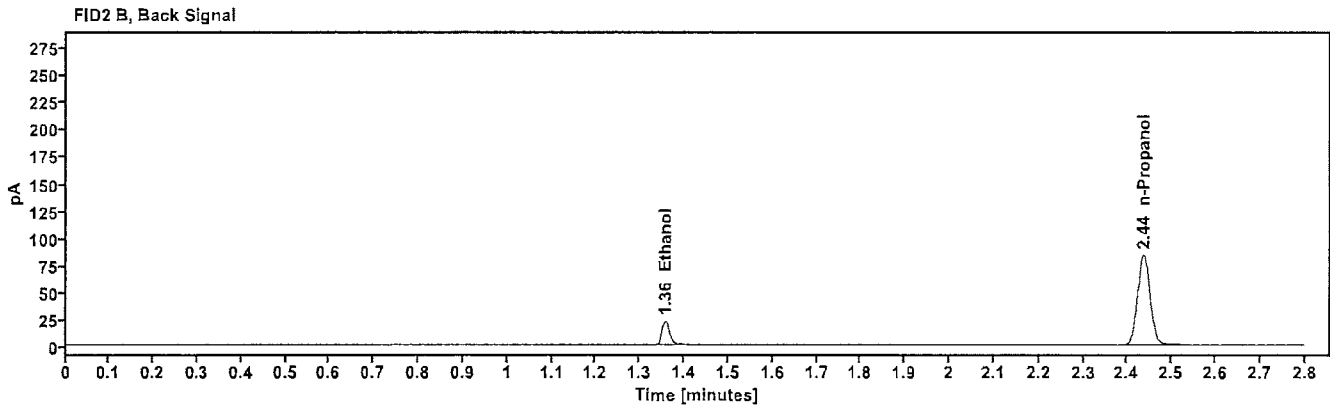
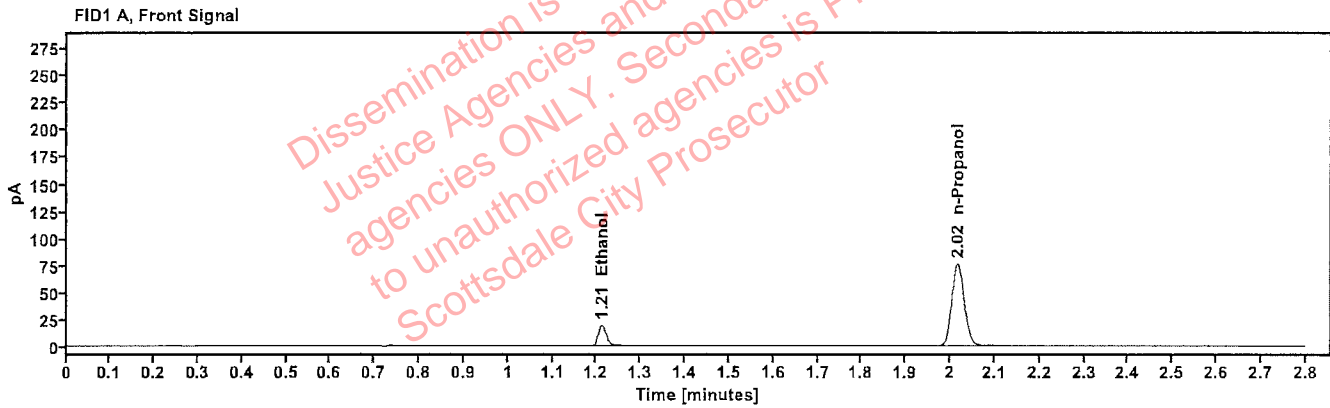


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0390	1.212	24.168
n-Propanol	-----	2.017	150.526

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.358	26.556
n-Propanol	2.436	164.830

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	4110320133/11	Vial:	44
Description:	0.199	LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 7:02:19 PM
Data file:	C:\Chem32\1\Data\31Oct23\44.D	Analyst:	Abbott JH

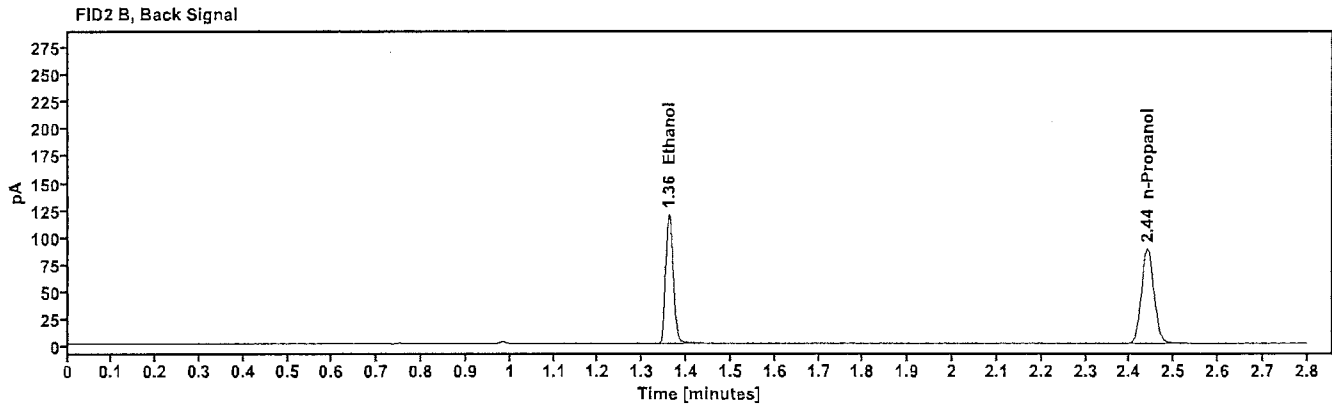
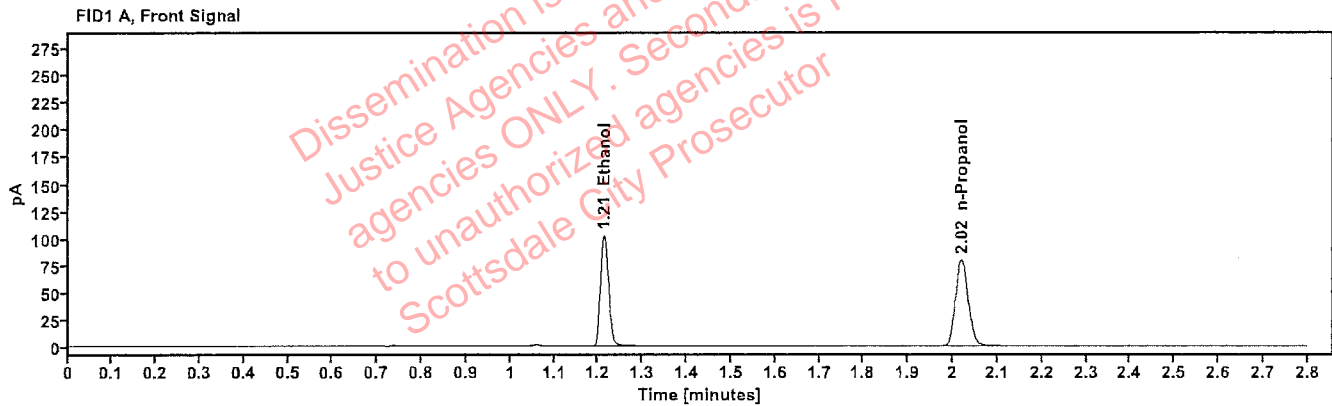


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2018	1.212	129.971
n-Propanol	-----	2.020	158.189

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.360	143.232
n-Propanol	2.440	173.320

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample: 092223AQ	Vial: 45
Description: Negative	LIMS ID:
Method: ethanol quant.M	Sequence: 31Oct23
Instrument: US14173023 CN14160045	Injection date: 10/31/2023 7:06:19 PM
Data file: C:\Chem32\1\Data\31Oct23\45.D	Analyst: Abbott

DA

Dissemination is restricted to Criminal Justice Agencies and authorized agencies ONLY. Secondary dissemination to unauthorized agencies is PROHIBITED. Scottsdale City Prosecutor

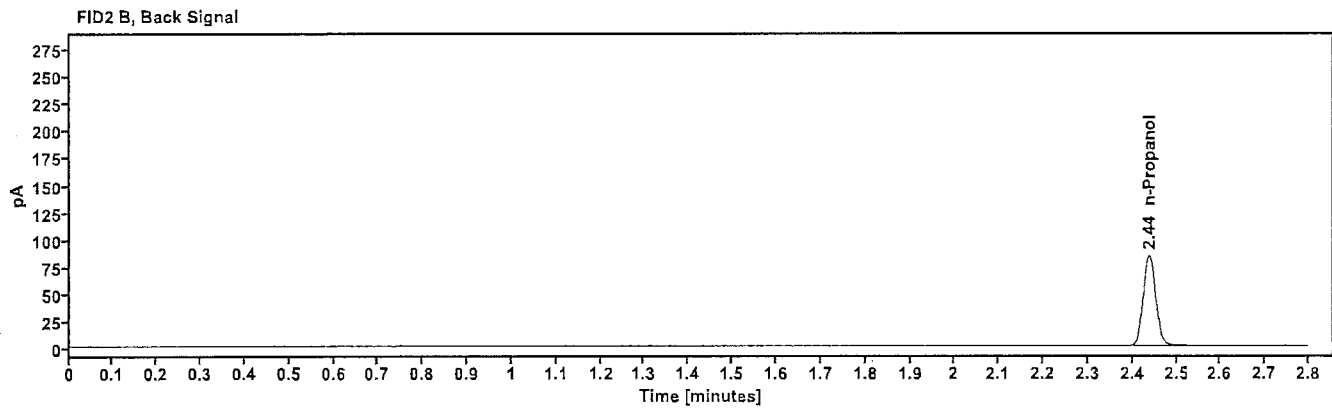
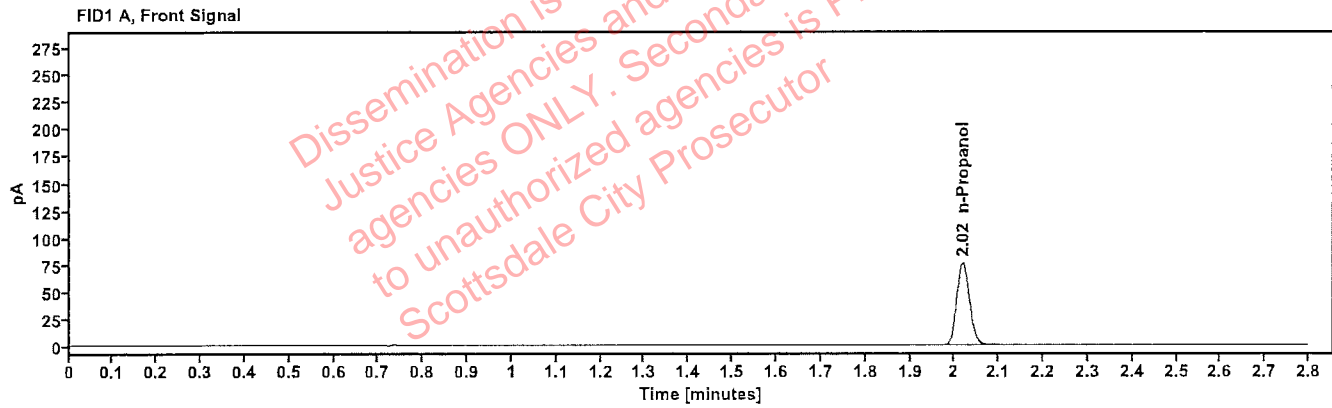


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
n-Propanol	-----	2.019	151.044

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
n-Propanol	2.438	165.366

Sequence Summary

Sequence name: 31Oct23 Instrument: US14173023 CN14160045 Analyst: Abbott

Vial	Sample	Description	Type	LIMS ID	Method
1	FN03122113	0.020 calibrator	Calibration		ethanol quant.M
2	FN11172002	0.100 calibrator	Calibration		ethanol quant.M
3	FN03132302	0.200 calibrator	Calibration		ethanol quant.M
4	FN03052102	0.400 calibrator	Calibration		ethanol quant.M
5	011823WB	Negative	Control		ethanol quant.M
6	050721MIX	Volatiles mix	Control		ethanol quant.M
7	30012020-A	0.400	Control		ethanol quant.M
8	14082019-B	0.040	Control		ethanol quant.M
9	4110320133/11	0.199	Control		ethanol quant.M
10	2300028-25		Sample		ethanol quant.M
11	2300028-25		Sample		ethanol quant.M
12	404-01J.J.		Sample		ethanol quant.M
13	404-01J.J.		Sample		ethanol quant.M
14	414-01B.P.		Sample		ethanol quant.M
15	414-01B.P.		Sample		ethanol quant.M
16	427-01B.L.		Sample		ethanol quant.M
17	427-01B.L.		Sample		ethanol quant.M
18	422-01Y.A.		Sample		ethanol quant.M
19	422-01Y.A.		Sample		ethanol quant.M
20	20012020-B	0.080	Control		ethanol quant.M
21	421-01D.G.		Sample		ethanol quant.M
22	421-01D.G.		Sample		ethanol quant.M
23	410-01A.V.		Sample		ethanol quant.M
24	410-01A.V.		Sample		ethanol quant.M
25	409-01W.P.		Sample		ethanol quant.M
26	409-01W.P.		Sample		ethanol quant.M
27	421-01A.A.		Sample		ethanol quant.M
28	421-01A.A.		Sample		ethanol quant.M
29	2321560-1		Sample		ethanol quant.M
30	2321560-1		Sample		ethanol quant.M
31	20012020-B	0.080	Control		ethanol quant.M
32	2313786-2		Sample		ethanol quant.M
33	2313786-2		Sample		ethanol quant.M
34	2317671-2		Sample		ethanol quant.M
35	2317671-2		Sample		ethanol quant.M
36	2321745-2		Sample		ethanol quant.M
37	2321745-2		Sample		ethanol quant.M
38	2321859-1		Sample		ethanol quant.M
39	2321859-1		Sample		ethanol quant.M
40	2321905-2		Sample		ethanol quant.M
41	2321905-2		Sample		ethanol quant.M
42	30012020-A	0.400	Control		ethanol quant.M
43	14082019-B	0.040	Control		ethanol quant.M
44	4110320133/11	0.199	Control		ethanol quant.M
45	092223AQ	Negative	Control		ethanol quant.M

Scottsdale Police Department Crime Laboratory Summary of Cases

SEQUENCE NAME: 31Oct23

ANALYST: Abbott *al*

Vials	Test 1 (g/dL)	Test 2 (g/dL)	Mean (g/dL)	Percent Difference*	Absolute Difference (g/dL)*
10 11	0.0537	0.0533	0.05350	0.37	0.00020
12 13	0.0689	0.0697	0.06930	0.58	0.00040
14 15	0.2356	0.2375	0.23655	0.40	0.00095
16 17	0.2071	0.2066	0.20685	0.12	0.00025
18 19	0.1561	0.1565	0.15630	0.13	0.00020
21 22	0.0477	0.0475	0.04760	0.21	0.00010
23 24	0.1891	0.1885	0.18880	0.16	0.00030
25 26	0.1867	0.1865	0.18660	0.05	0.00010
27 28	0.2396	0.2422	0.24090	0.54	0.00130
29 30	0.1082	0.1073	0.10775	0.42	0.00045
32 33	0.1417	0.1426	0.14215	0.32	0.00045
34 35	0.2215	0.2223	0.22190	0.18	0.00040
36 37	0.2774	0.2769	0.27715	0.09	0.00025
38 39	0.1753	0.1748	0.17505	0.14	0.00025
40 41	0.2121	0.2146	0.21335	0.59	0.00125

*Calculated differences are differences from the mean of the two results.

Scottsdale Forensic Lab

Blood Alcohol Pipetting Log

ANALYST: Abbott



SEQUENCE: 310Oct23

Instrument Position	Headspace Vial 1	Headspace Vial 2	Blood Tube	Barcode Match
Vials 10 and 11	2300028-25	2300028-25	2300028-25	Yes
Vials 12 and 13	404-01J.J.	404-01J.J.	404-01J.J.	Yes
Vials 14 and 15	414-01B.P.	414-01B.P.	414-01B.P.	Yes
Vials 16 and 17	427-01B.L.	427-01B.L.	427-01B.L.	Yes
Vials 18 and 19	422-01Y.A.	422-01Y.A.	422-01Y.A.	Yes
Vials 21 and 22	421-01D.G.	421-01D.G.	421-01D.G.	Yes
Vials 23 and 24	410-01A.V.	410-01A.V.	410-01A.V.	Yes
Vials 25 and 26	409-01W.P.	409-01W.P.	409-01W.P.	Yes
Vials 27 and 28	421-01A.A.	421-01A.A.	421-01A.A.	Yes
Vials 29 and 30	2321560-1	2321560-1	2321560-1	Yes
Vials 32 and 33	2313786-2	2313786-2	2313786-2	Yes
Vials 34 and 35	2317671-2	2317671-2	2317671-2	Yes
Vials 36 and 37	2321745-2	2321745-2	2321745-2	Yes
Vials 38 and 39	2321859-1	2321859-1	2321859-1	Yes
Vials 40 and 41	2321905-2	2321905-2	2321905-2	Yes

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2300028-25	Vial:	10
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 4:44:48 PM
Data file:	C:\Chem32\1\Data\31Oct23\10.D	Analyst:	Abbott

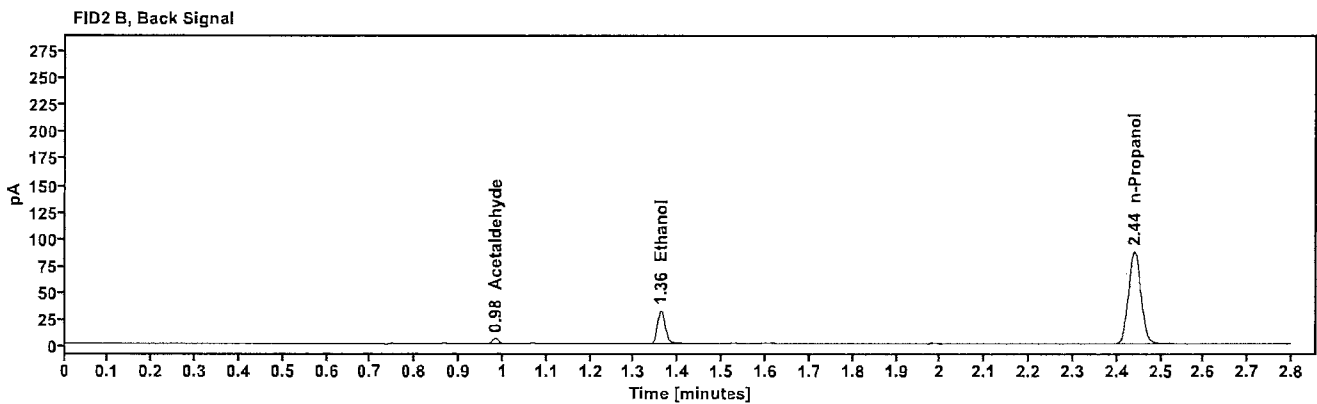
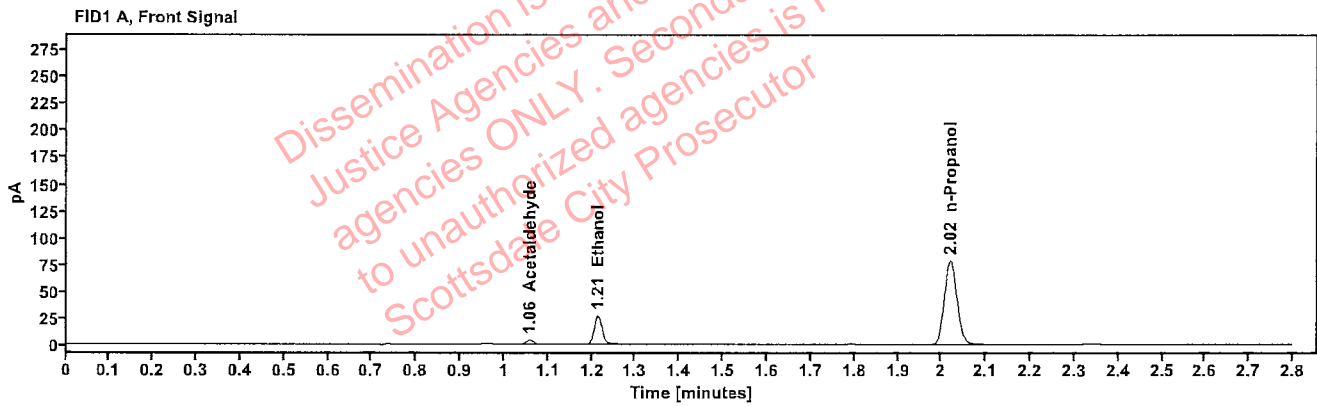


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
Acetaldehyde	-----	1.058	4.312
>Ethanol	0.0537	1.214	34.327
n-Propanol	-----	2.019	155.788

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Acetaldehyde	0.983	4.870
Ethanol	1.361	38.167
n-Propanol	2.440	171.178

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2300028-25	Vial:	11
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 4:48:48 PM
Data file:	C:\Chem32\1\Data\31Oct23\11.D	Analyst:	Abbott

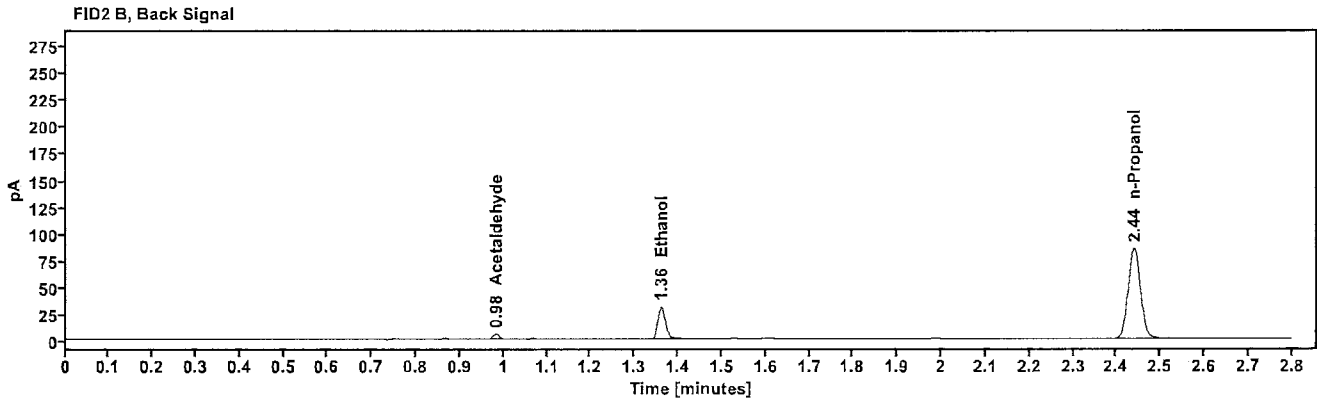
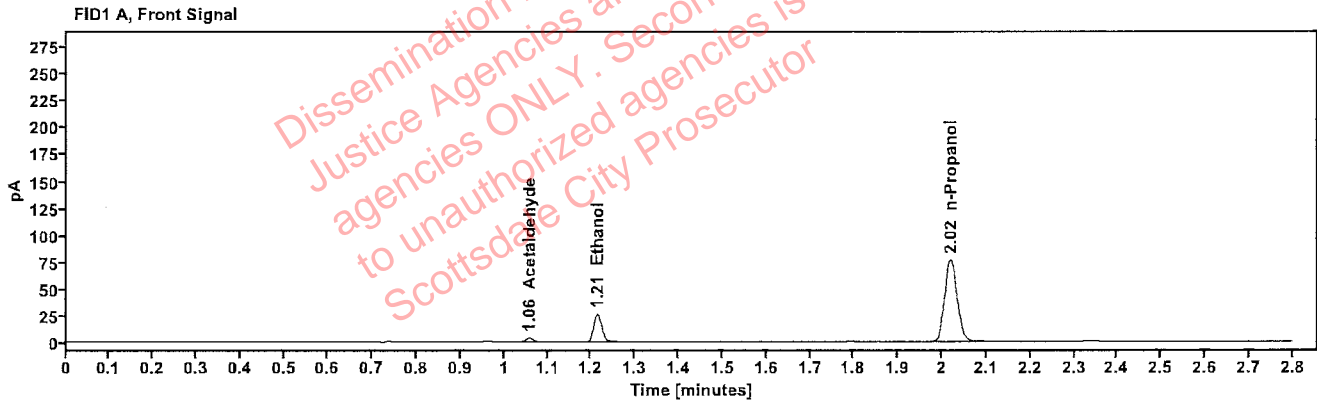


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
Acetaldehyde	-----	1.058	4.249
>Ethanol	0.0533	1.214	33.280
n-Propanol	-----	2.019	152.165

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Acetaldehyde	0.983	4.805
Ethanol	1.361	37.133
n-Propanol	2.440	167.388

[REDACTED]: Abbott, Lori - (01-Nov-2023)

Case: [REDACTED] User: LAbbott 11/1/2023

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	404-01J.J.	Vial:	12
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 4:52:48 PM
Data file:	C:\Chem32\1\Data\31Oct23\12.D	Analyst:	Abbott

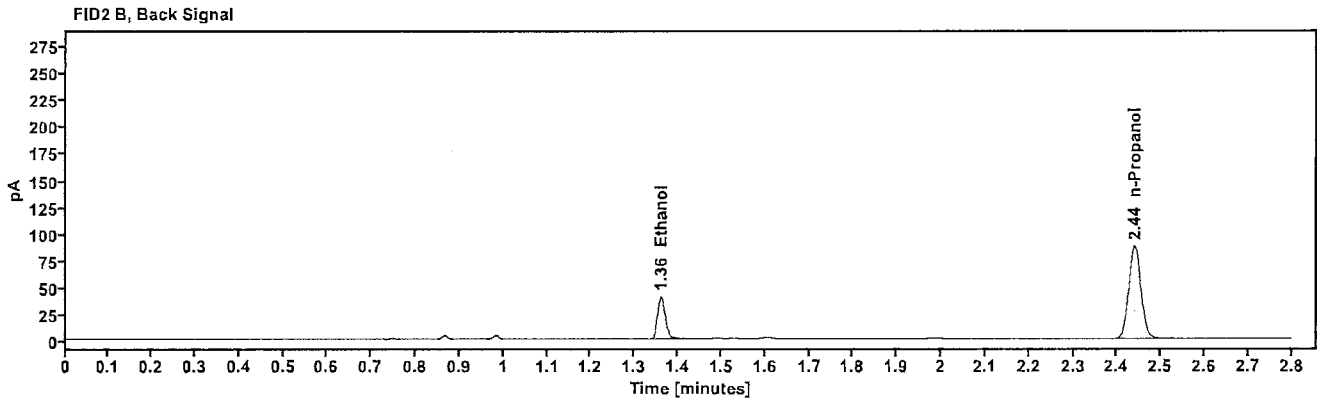
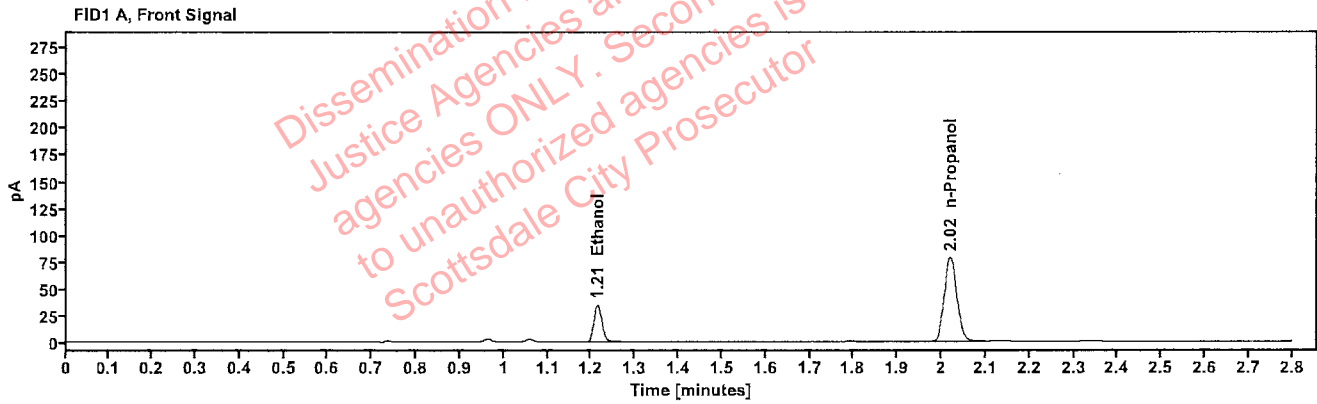


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0689	1.213	44.211
n-Propanol	---	2.019	156.912

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	48.777
n-Propanol	2.440	171.728

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	404-01J.J.	Vial:	13
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 4:57:03 PM
Data file:	C:\Chem32\1\Data\31Oct23\13.D	Analyst:	Abbott

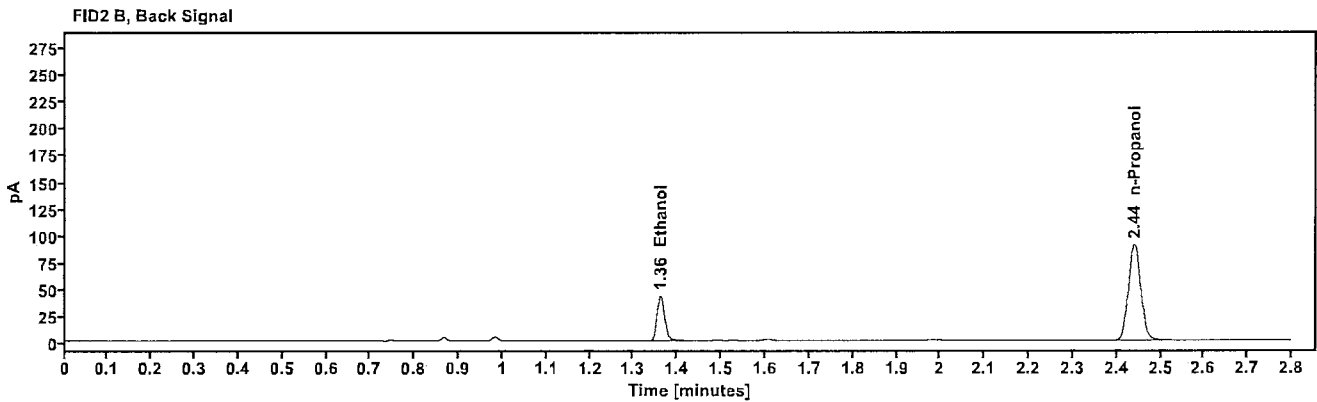
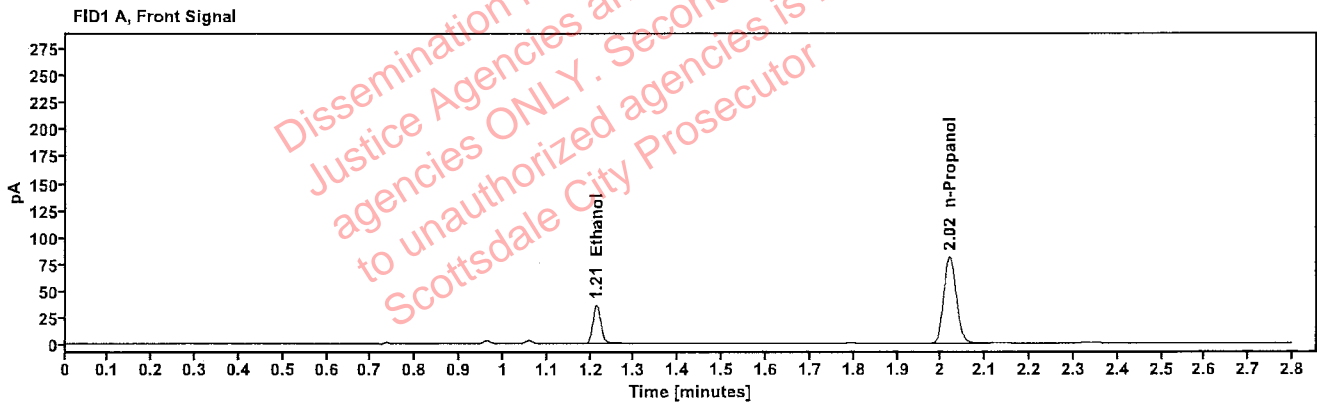


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0697	1.214	46.168
n-Propanol	-----	2.020	161.989

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	50.960
n-Propanol	2.440	177.216

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	414-01B.P.	Vial:	14
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:01:03 PM
Data file:	C:\Chem32\1\Data\31Oct23\14.D	Analyst:	Abbott

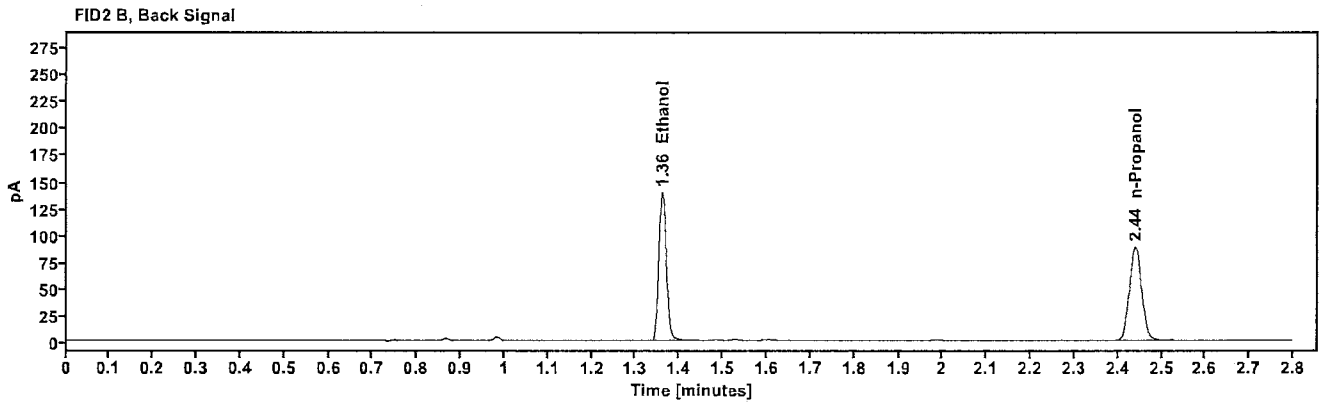
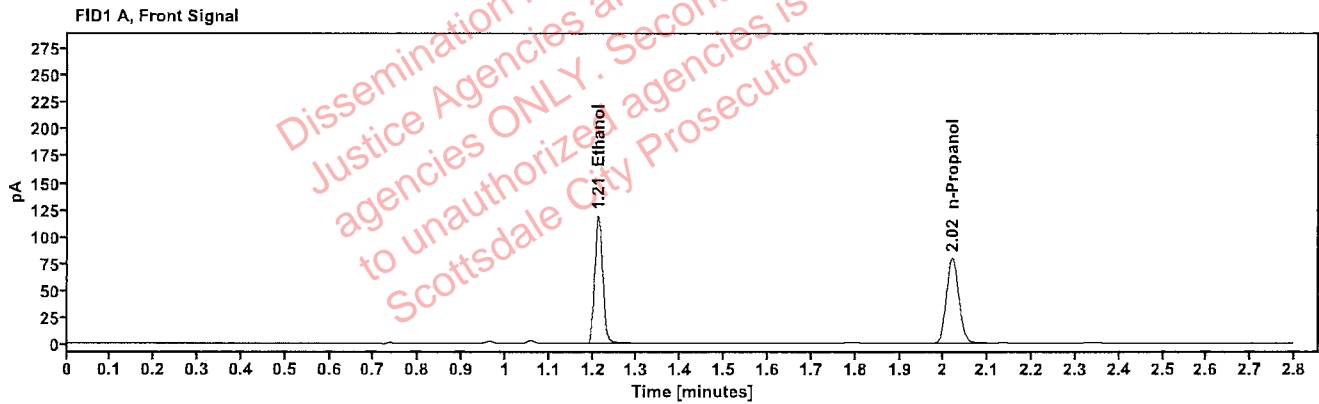


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2356	1.212	151.344
n-Propanol	---	2.019	157.815

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.360	166.727
n-Propanol	2.440	173.274

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	414-01B.P.	Vial:	15
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:05:04 PM
Data file:	C:\Chem32\1\Data\31Oct23\15.D	Analyst:	Abbott

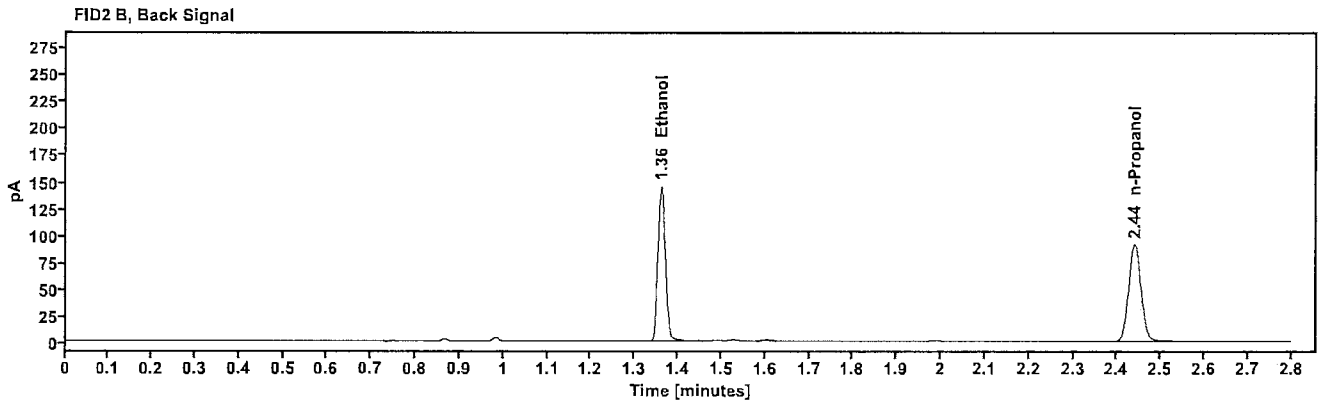
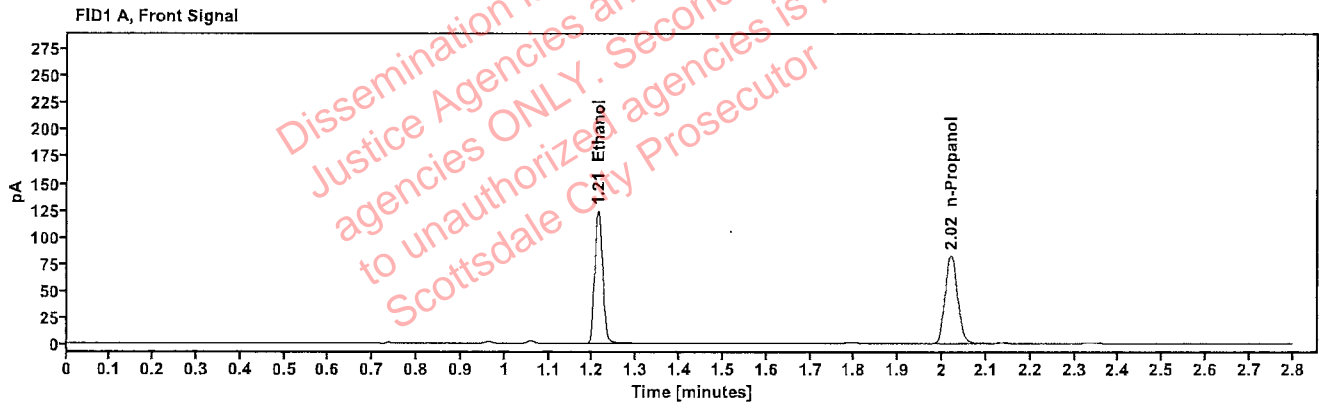


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2375	1.213	157.964
n-Propanol	---	2.020	163.428

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	174.138
n-Propanol	2.440	179.534

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	427-01B.L.	Vial:	16
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:09:04 PM
Data file:	C:\Chem32\1\Data\31Oct23\16.D	Analyst:	Abbott

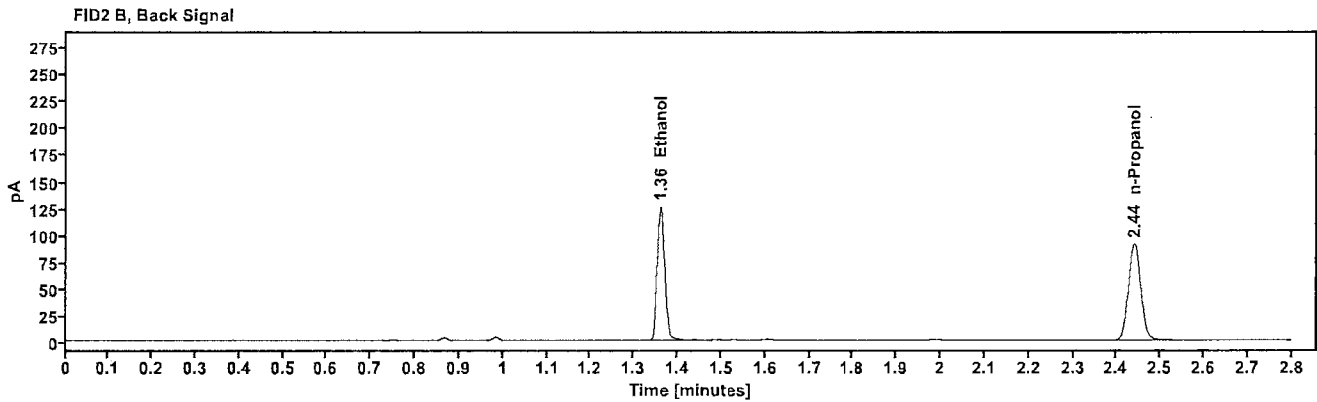
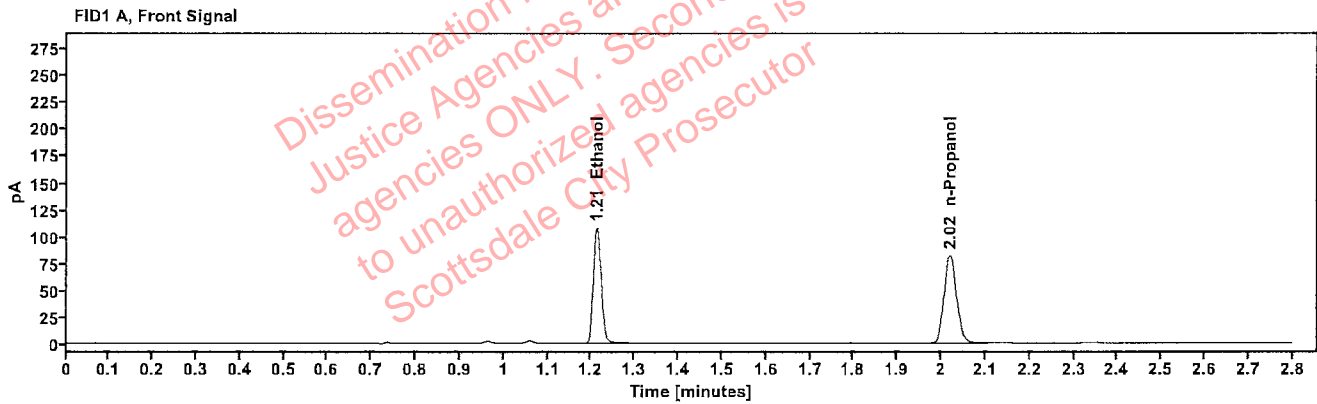


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2071	1.213	137.308
n-Propanol	-----	2.020	162.852

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	151.461
n-Propanol	2.440	178.728

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	427-01B.L.	Vial:	17
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:13:03 PM
Data file:	C:\Chem32\1\Data\31Oct23\17.D	Analyst:	Abbott

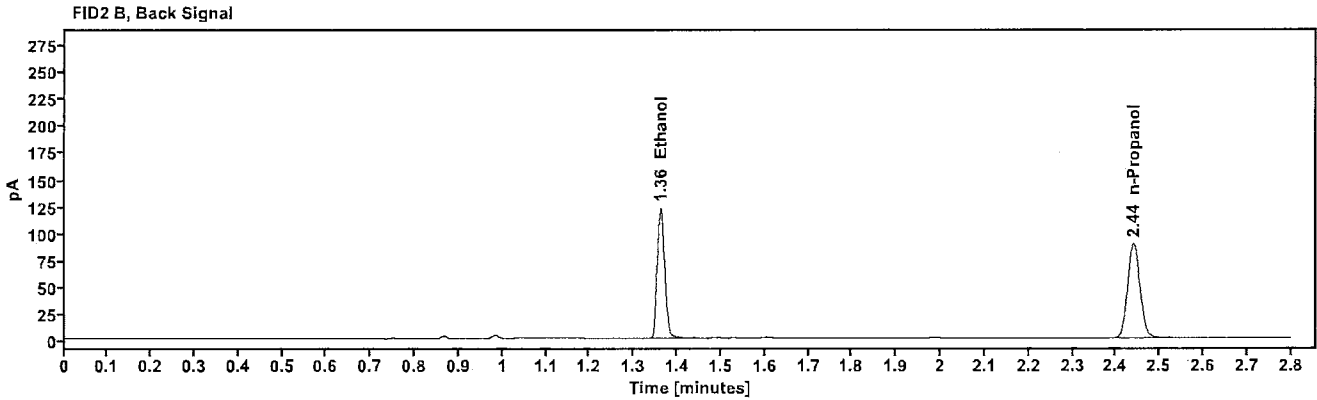
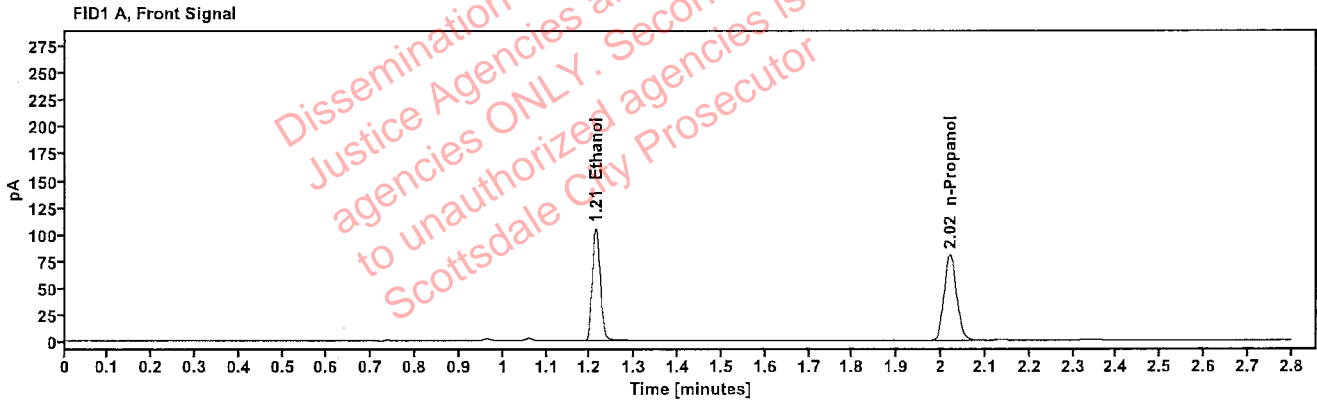


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2066	1.213	133.709
n-Propanol	—	2.020	158.999

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	147.443
n-Propanol	2.440	174.596

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	422-01Y.A.	Vial:	18
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:17:03 PM
Data file:	C:\Chem32\1\Data\31Oct23\18.D	Analyst:	Abbott

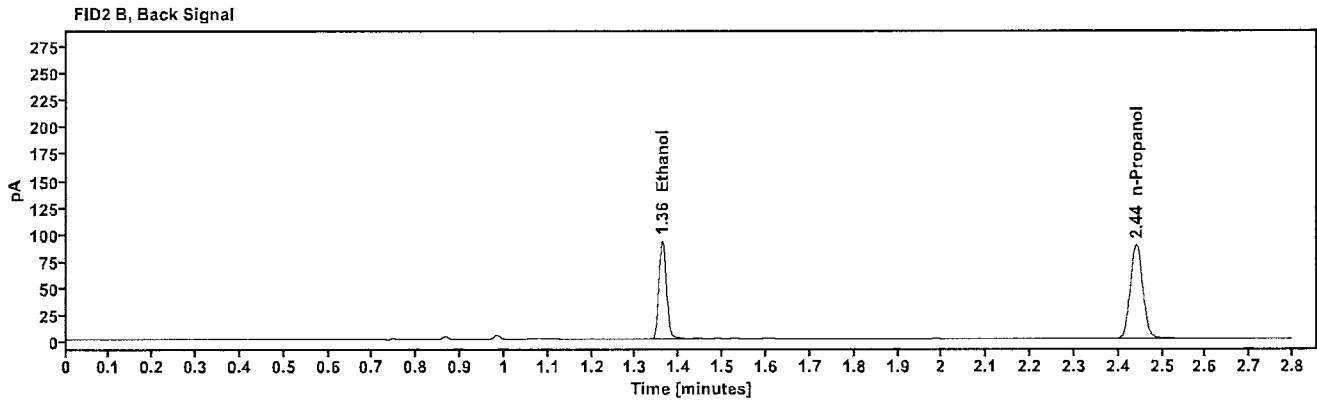
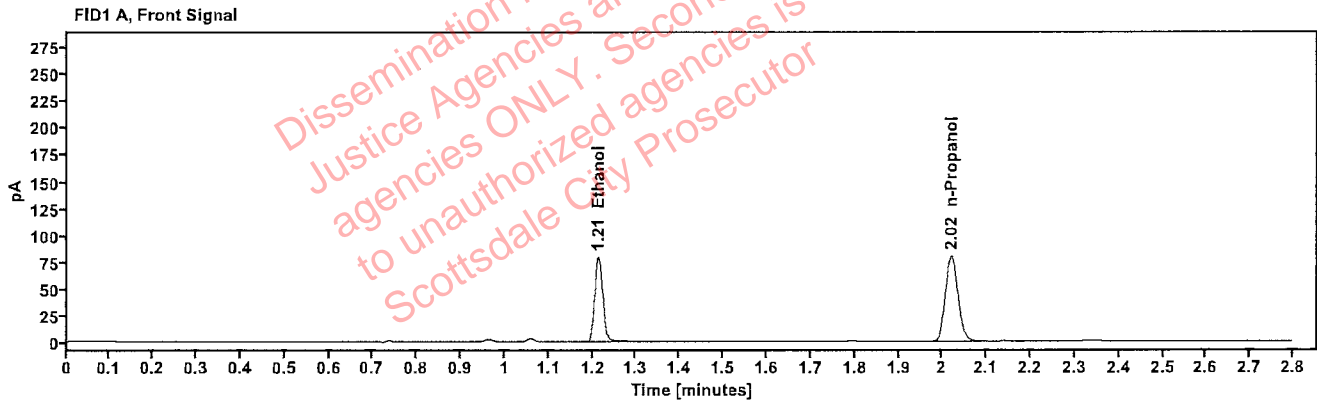


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1561	1.213	100.897
n-Propanol	-----	2.020	158.626

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	111.129
n-Propanol	2.441	174.317

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	422-01Y.A.	Vial:	19
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:21:18 PM
Data file:	C:\Chem32\1\Data\31Oct23\19.D	Analyst:	Abbott

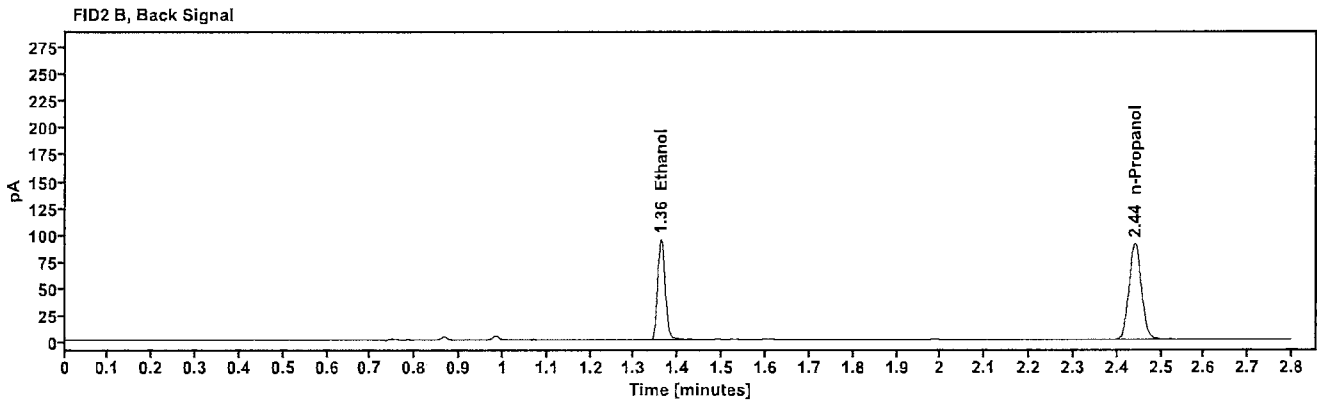
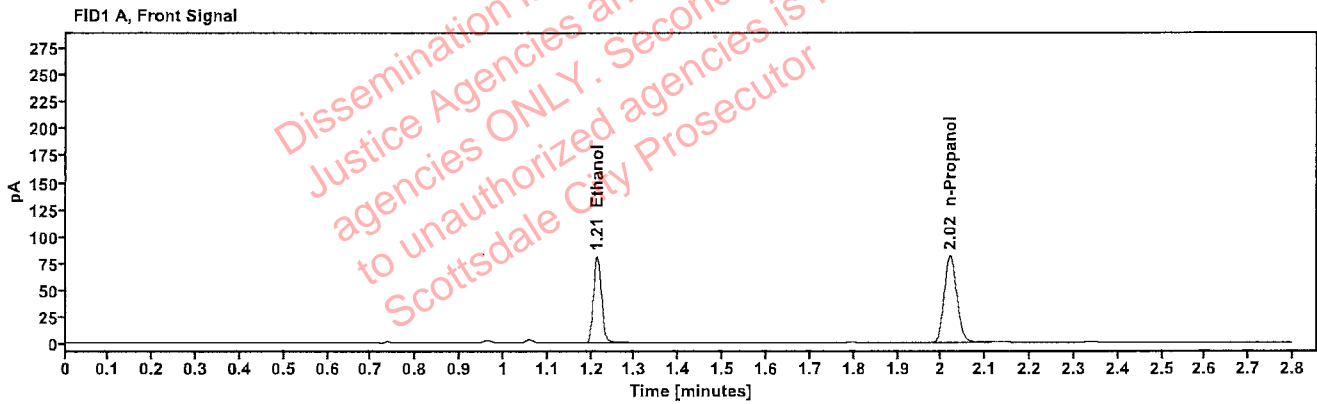


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1565	1.213	103.033
n-Propanol	-----	2.020	161.561

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	113.111
n-Propanol	2.440	177.416

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	421-01D.G.	Vial:	21
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:29:19 PM
Data file:	C:\Chem3211\Data\31Oct23\21.D	Analyst:	Abbott

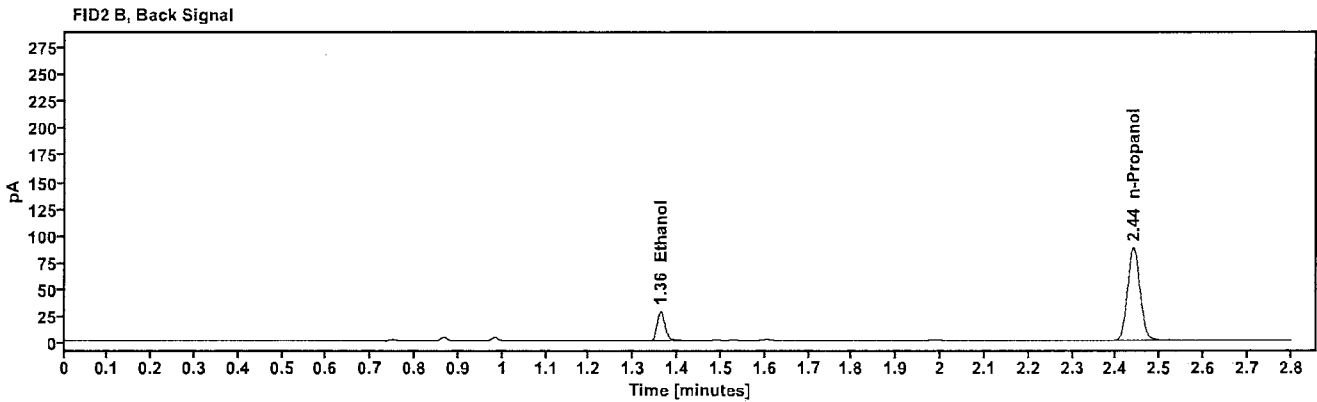
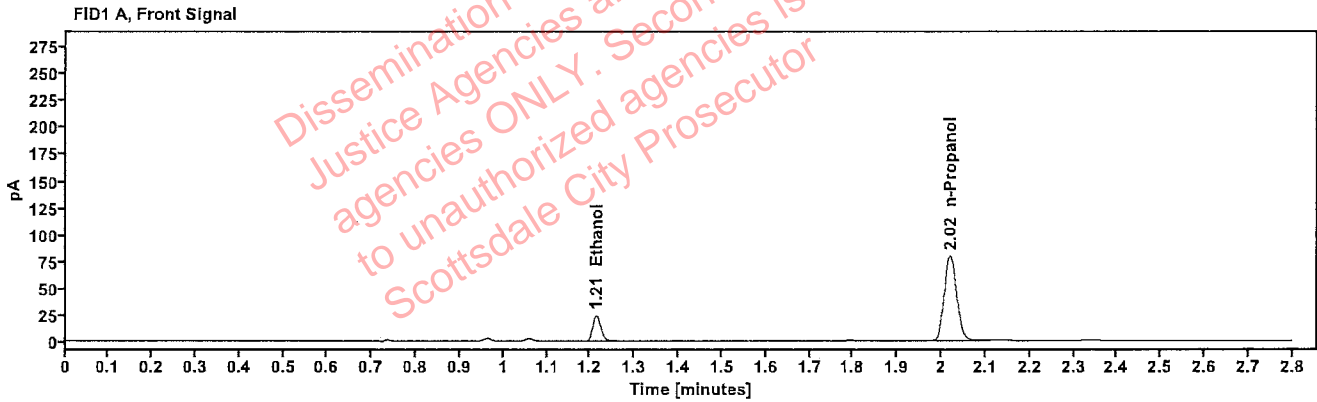


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0477	1.214	30.826
n-Propanol	-----	2.019	157.345

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	34.371
n-Propanol	2.440	172.809

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	421-01D.G.	Vial:	22
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:33:19 PM
Data file:	C:\Chem32\1\Data\31Oct23\22.D	Analyst:	Abbott

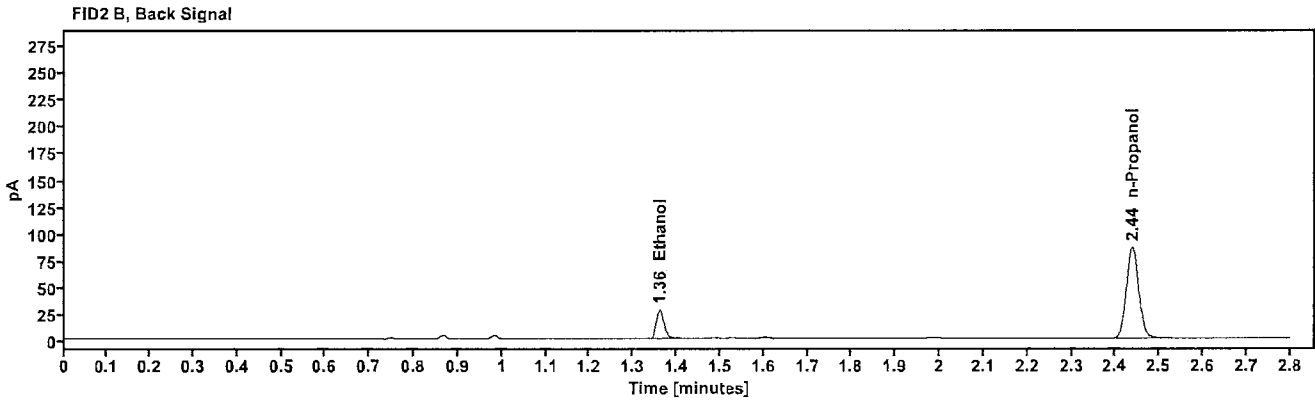
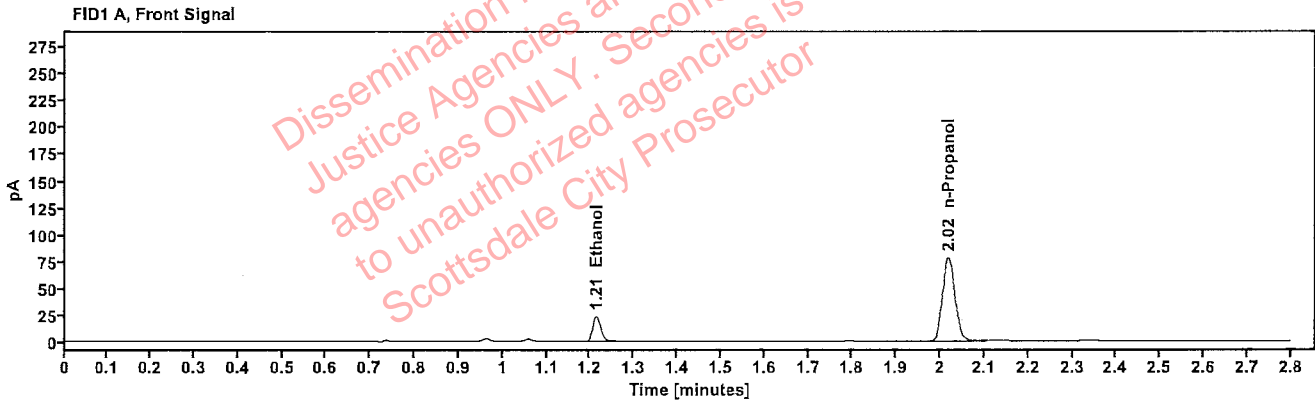


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.0475	1.214	30.202
n-Propanol	-----	2.018	154.855

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	33.695
n-Propanol	2.438	169.315

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	410-01A.V.	Vial:	23
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:37:19 PM
Data file:	C:\Chem32\1\Data\31Oct23\23.D	Analyst:	Abbott

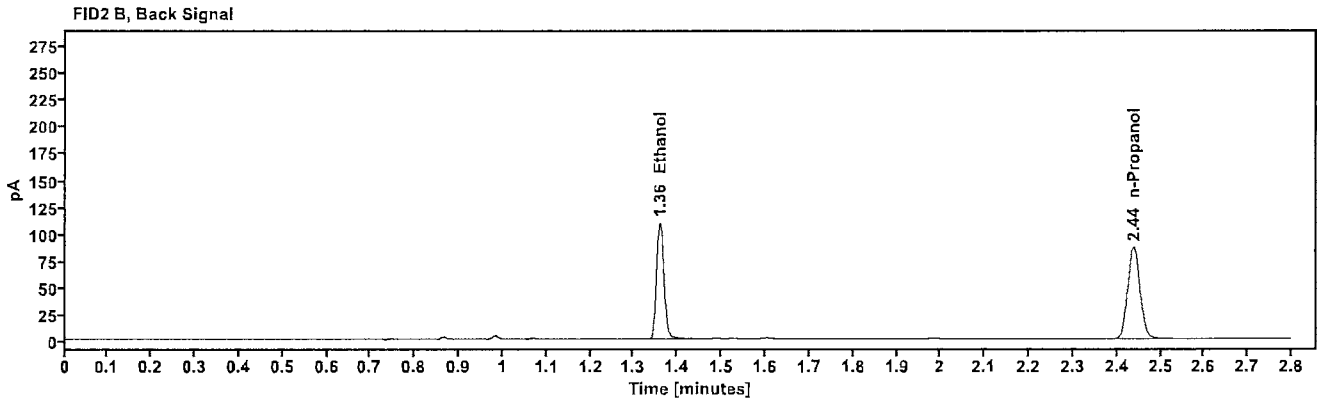
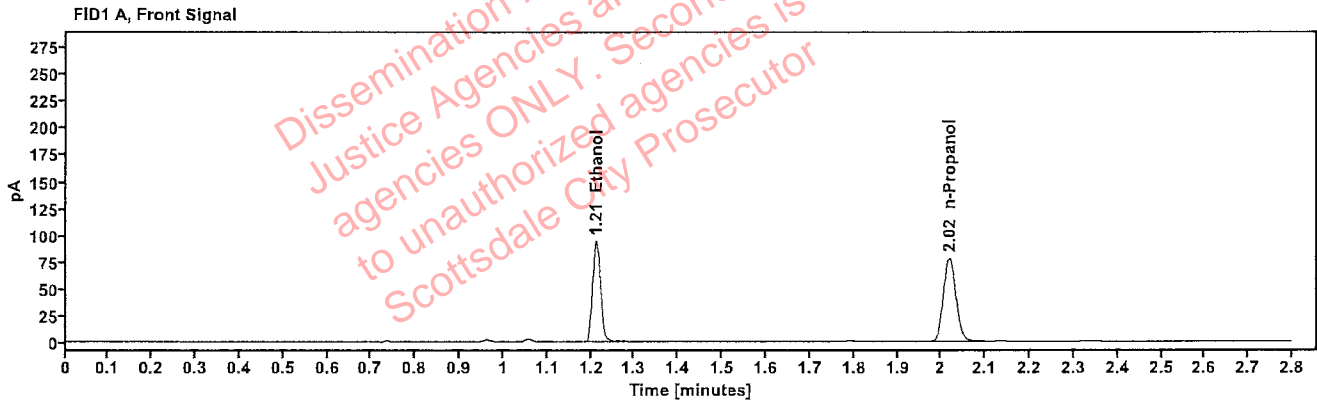


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1891	1.212	119.210
n-Propanol	-----	2.018	154.827

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.360	131.076
n-Propanol	2.439	169.871

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	410-01A.V.	Vial:	24
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:41:18 PM
Data file:	C:\Chem32\1\Data\31Oct23\24.D	Analyst:	Abbott

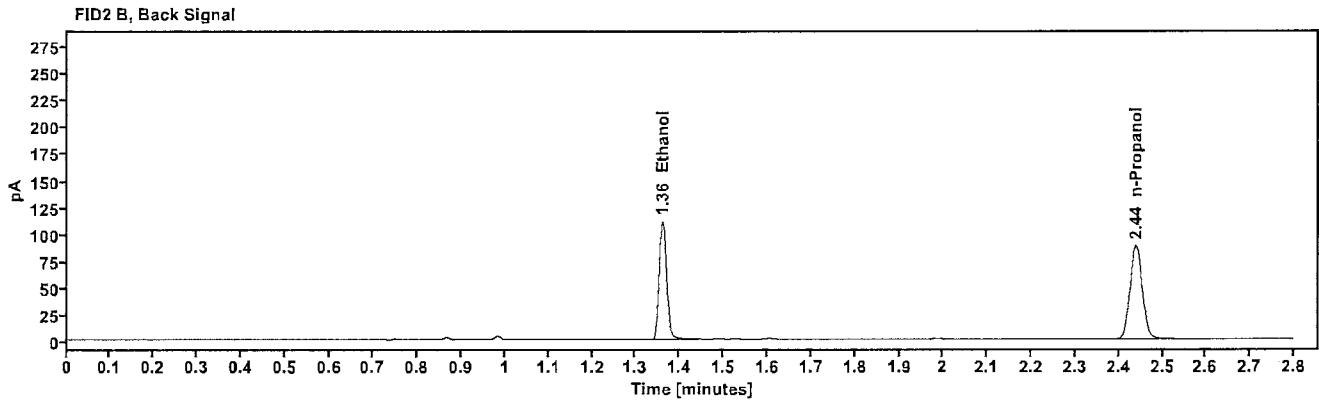
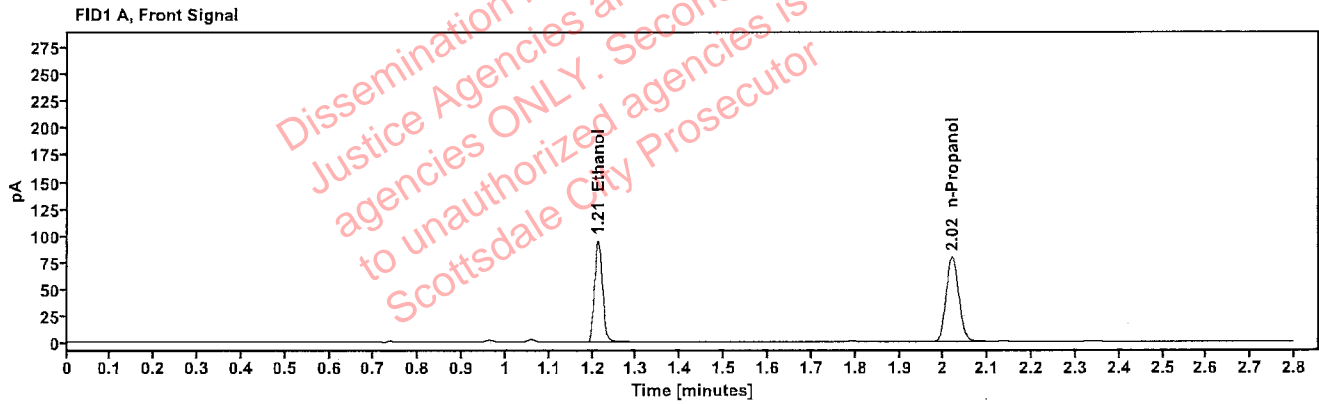


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1885	1.212	120.761
n-Propanol	-----	2.019	157.357

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.360	133.012
n-Propanol	2.439	172.841

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	409-01W.P.	Vial:	25
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:45:33 PM
Data file:	C:\Chem32\1\Data\31Oct23\25.D	Analyst:	Abbott

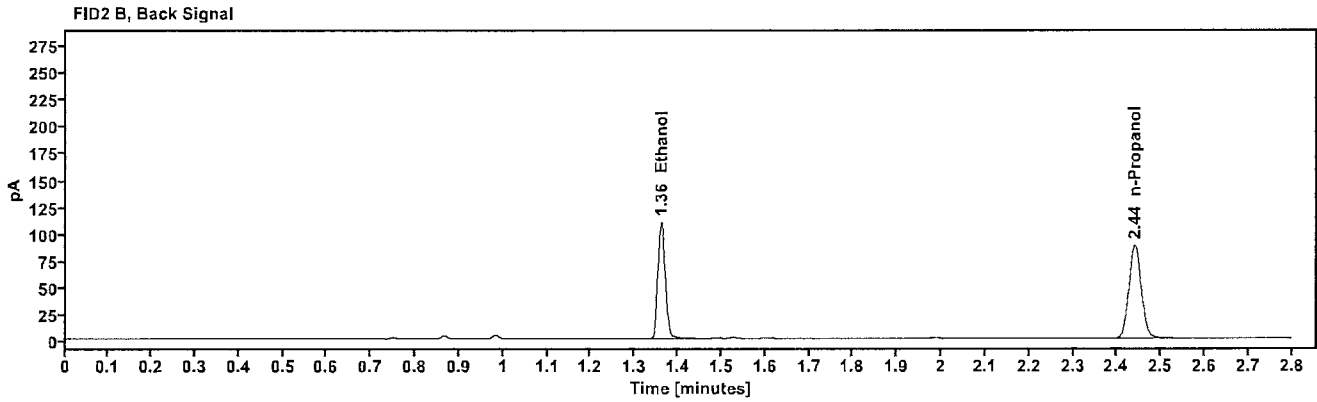
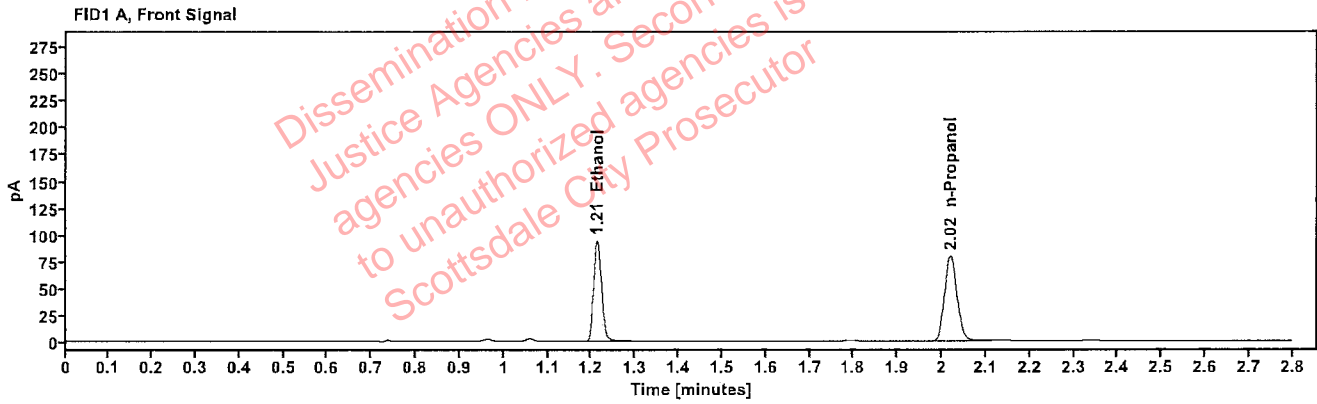


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1867	1.213	119.820
n-Propanol	-----	2.020	157.641

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	131.838
n-Propanol	2.440	172.873

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	409-01W.P.	Vial:	26
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:49:33 PM
Data file:	C:\Chem32\1\Data\31Oct23\26.D	Analyst:	Abbott

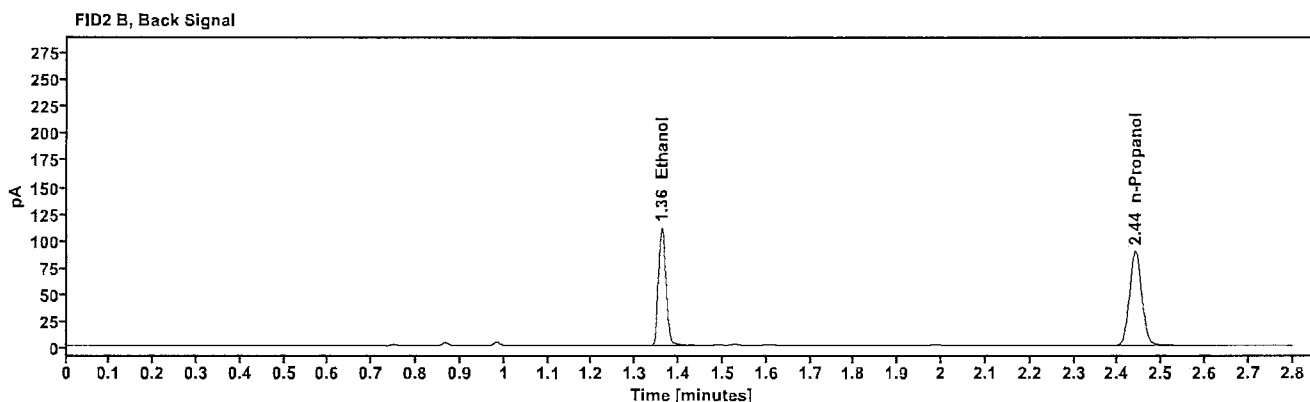
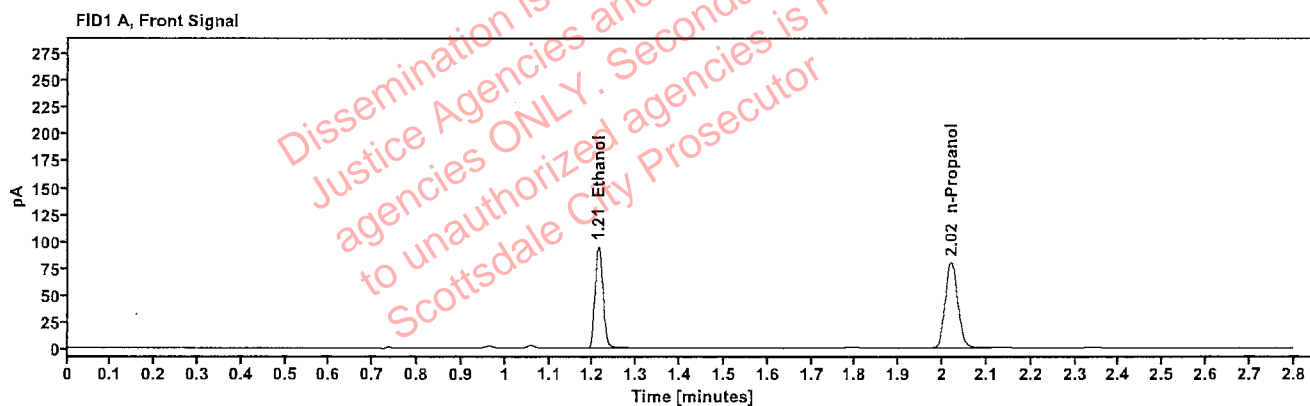


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1865	1.213	120.777
n-Propanol	---	2.020	159.051

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	133.090
n-Propanol	2.440	174.484

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	421-01A.A.	Vial:	27
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:53:33 PM
Data file:	C:\Chem32\1\Data\31Oct23\27.D	Analyst:	Abbott

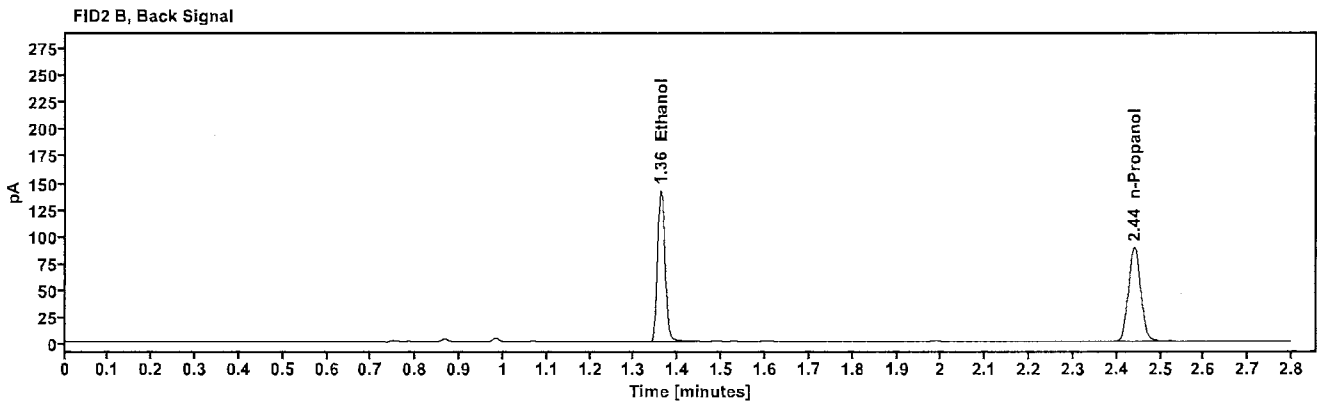
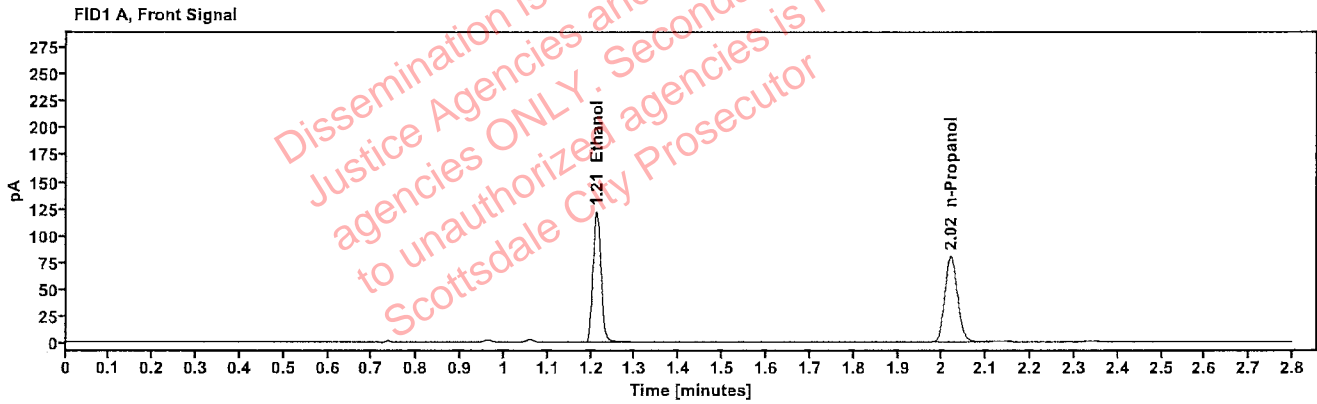


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2396	1.212	154.950
n-Propanol	---	2.020	158.895

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	170.716
n-Propanol	2.440	174.451

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	421-01A.A.	Vial:	28
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 5:57:33 PM
Data file:	C:\Chem32\1\Data\31Oct23\28.D	Analyst:	Abbott

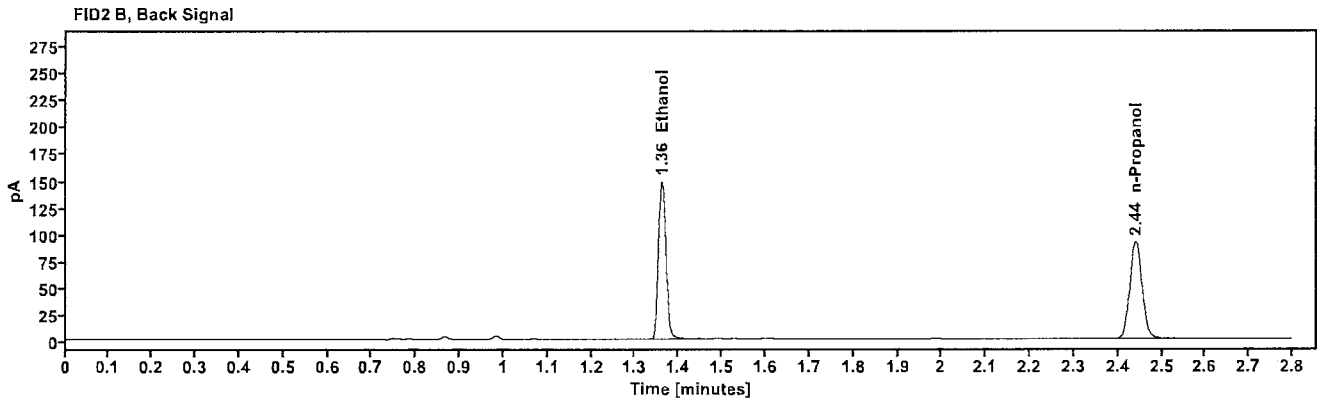
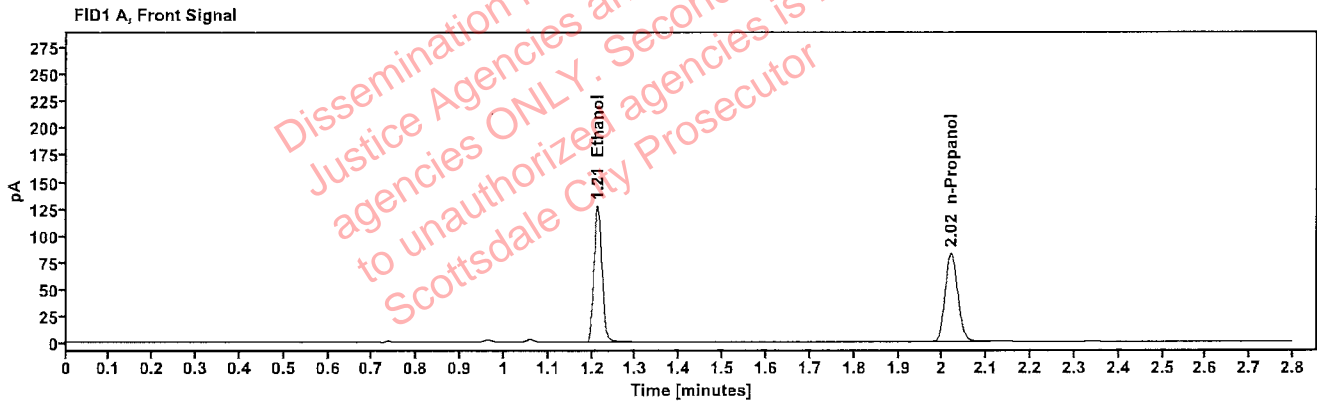


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2422	1.213	161.771
n-Propanol	-----	2.020	164.109

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	178.427
n-Propanol	2.440	180.174

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321560-1	Vial:	29
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:01:33 PM
Data file:	C:\Chem32\1\Data\31Oct23\29.D	Analyst:	Abbott

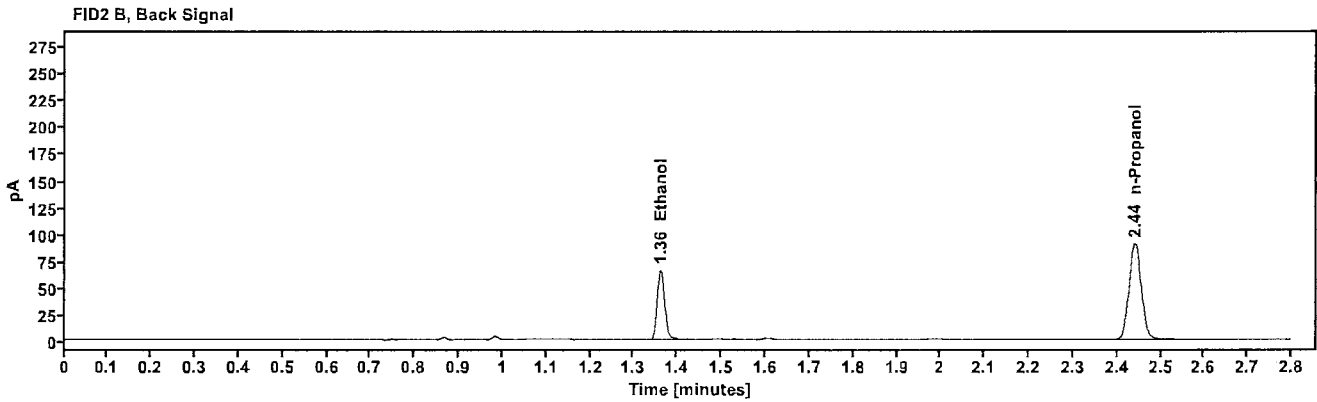
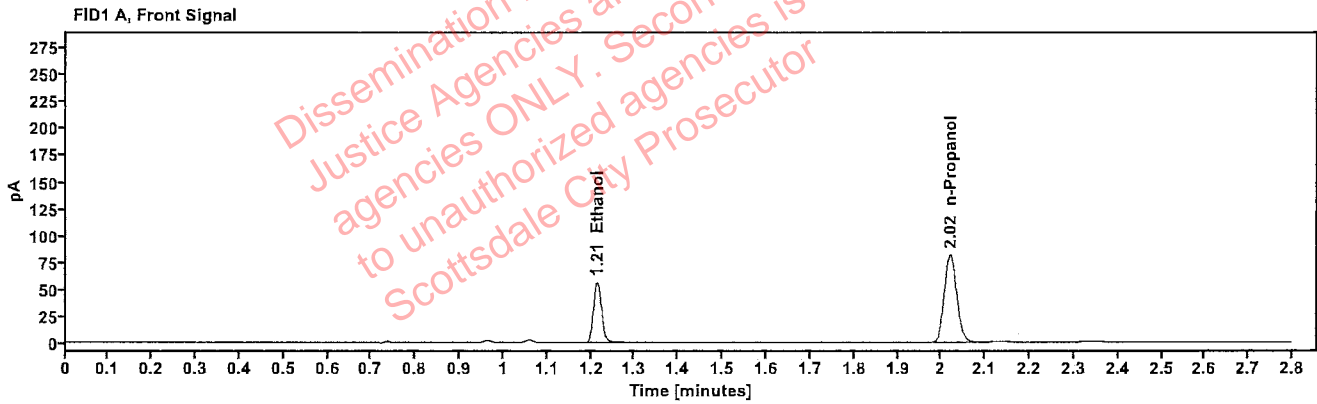


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1082	1.214	71.907
n-Propanol	---	2.020	162.905

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	79.289
n-Propanol	2.441	178.216

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321560-1	Vial:	30
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:05:33 PM
Data file:	C:\Chem32\1\Data\31Oct23\30.D	Analyst:	Abbott

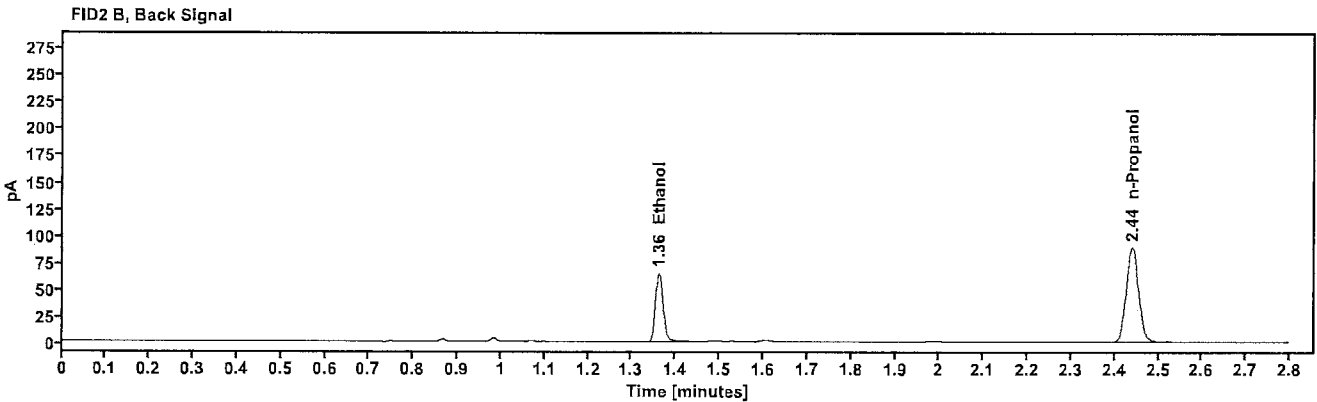
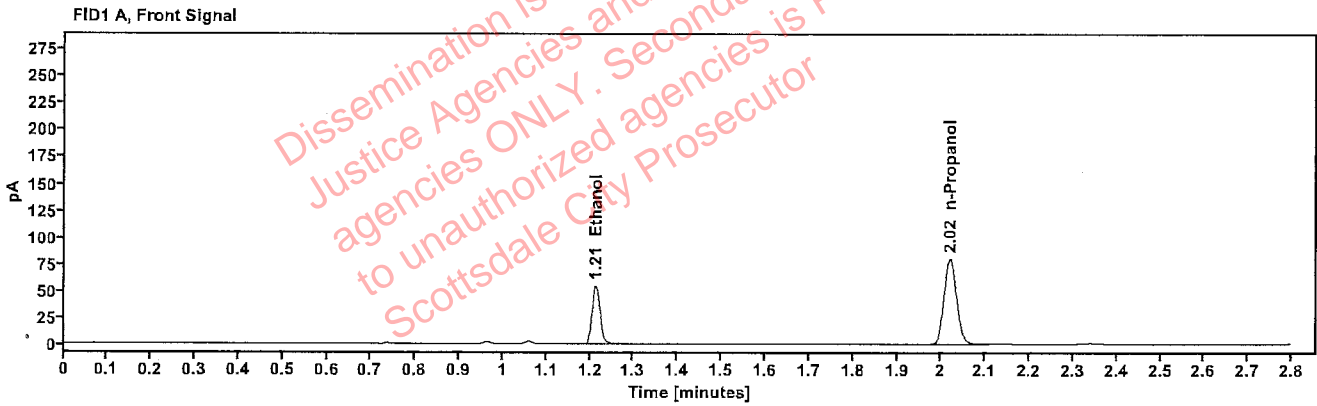


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1073	1.213	70.018
n-Propanol	---	2.020	159.954

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	77.285
n-Propanol	2.440	175.015

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2313786-2	Vial:	32
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:13:48 PM
Data file:	C:\Chem32\1\Data\31Oct23\32.D	Analyst:	Abbott

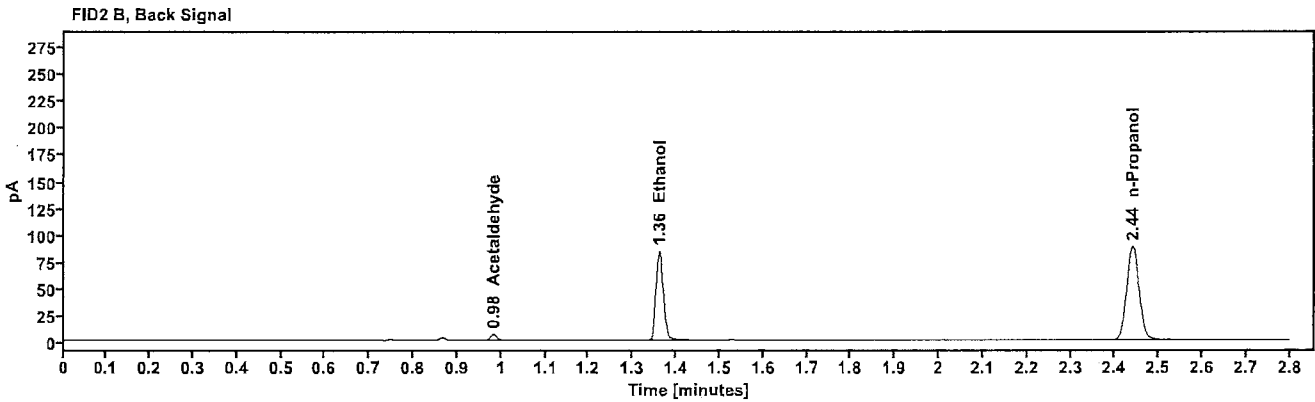
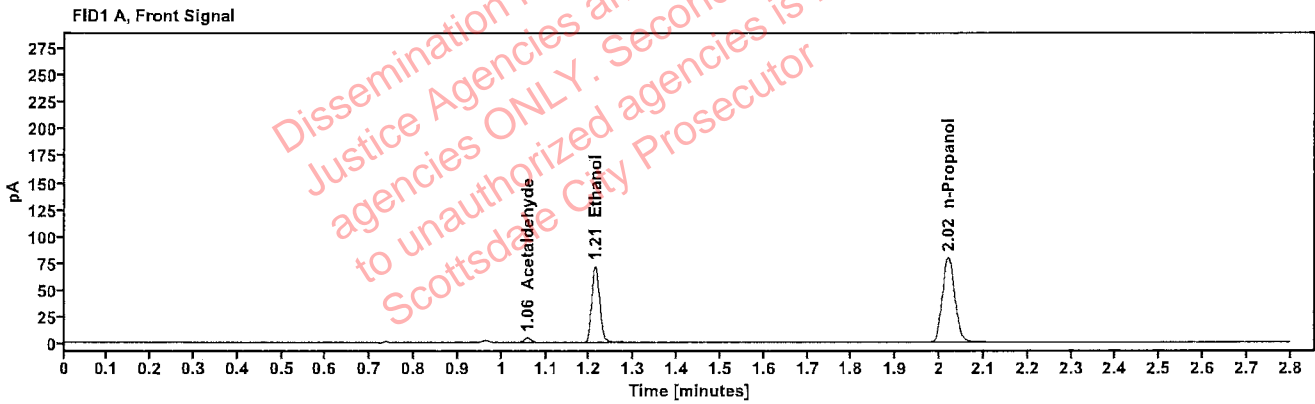


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
Acetaldehyde	-----	1.058	4.740
>Ethanol	0.1417	1.213	91.195
n-Propanol	-----	2.020	157.942

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Acetaldehyde	0.983	5.350
Ethanol	1.361	99.998
n-Propanol	2.440	173.041

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2313786-2	Vial:	33
Description:	██████████	LIMS ID:	██████████
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:17:49 PM
Data file:	C:\Chem3211\Data\31Oct23\33.D	Analyst:	Abbott

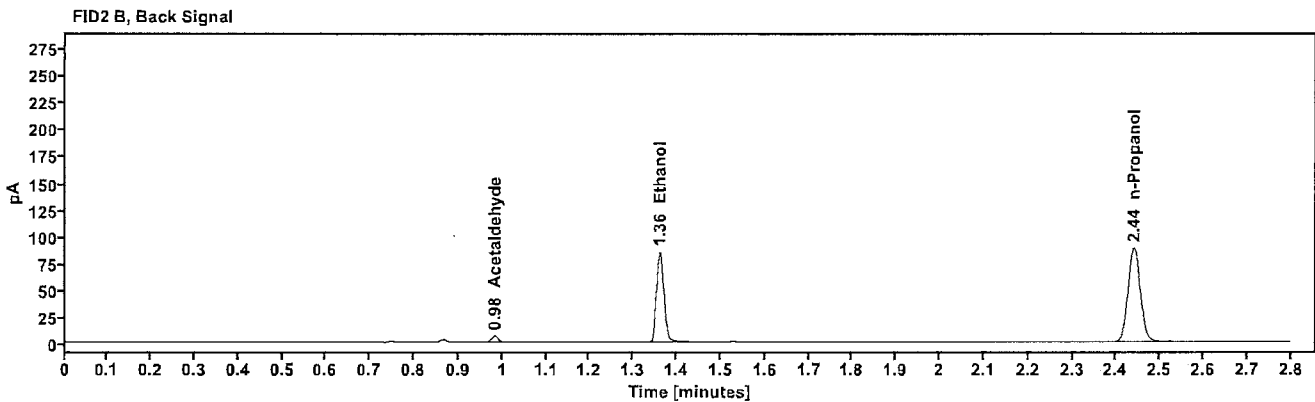
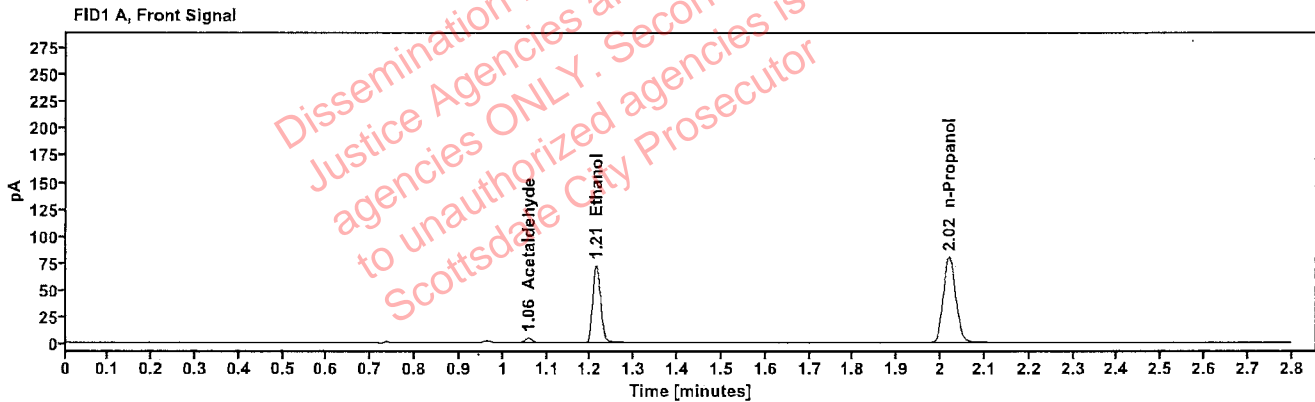


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
Acetaldehyde	-----	1.058	4.761
>Ethanol	0.1426	1.213	92.242
n-Propanol	-----	2.020	158.693

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Acetaldehyde	0.983	5.388
Ethanol	1.361	101.250
n-Propanol	2.440	173.802

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2317671-2	Vial:	34
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:21:48 PM
Data file:	C:\Chem32\1\Data\31Oct23\34.D	Analyst:	Abbott

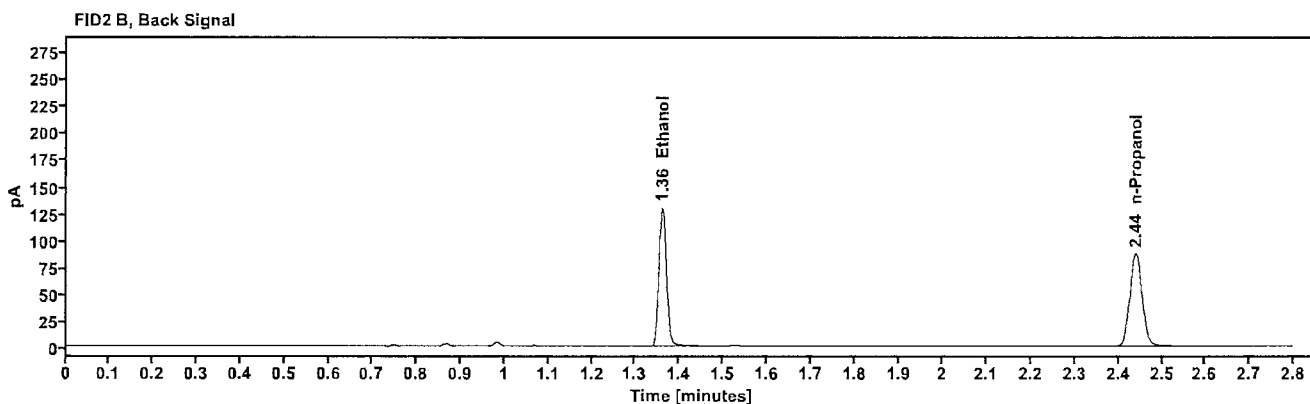
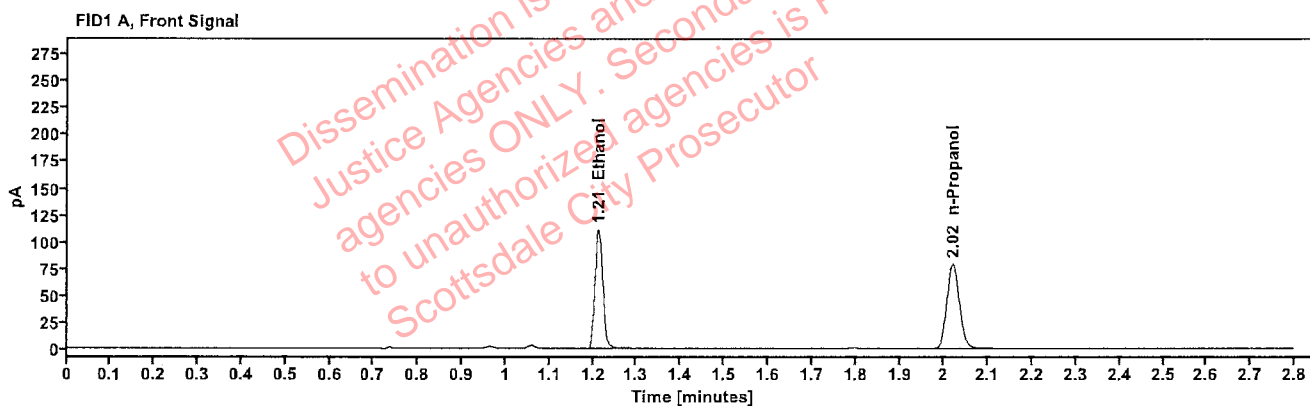


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2215	1.212	141.647
n-Propanol	---	2.020	157.094

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.360	155.996
n-Propanol	2.440	172.147

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2317671-2	Vial:	35
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:25:48 PM
Data file:	C:\Chem32\1\Data\31Oct23\35.D	Analyst:	Abbott

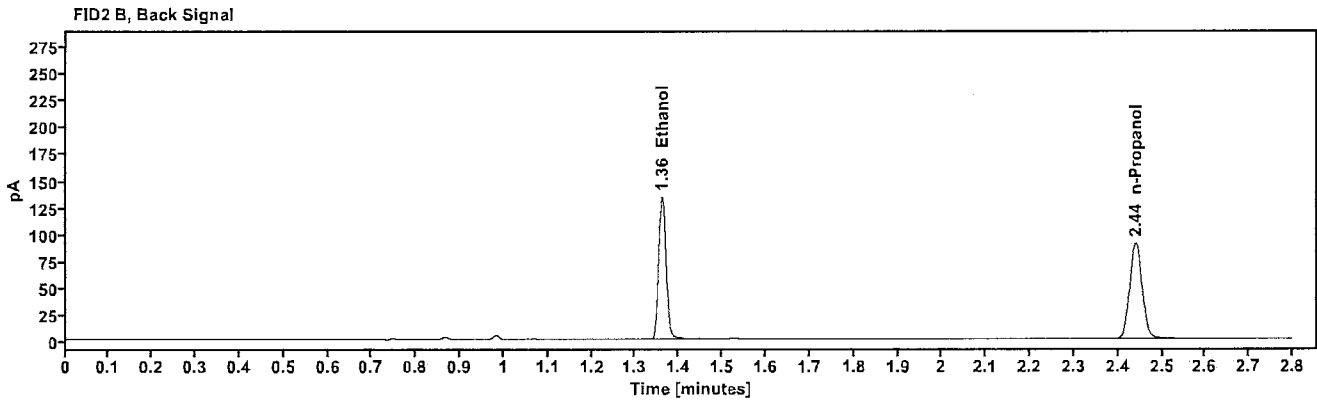
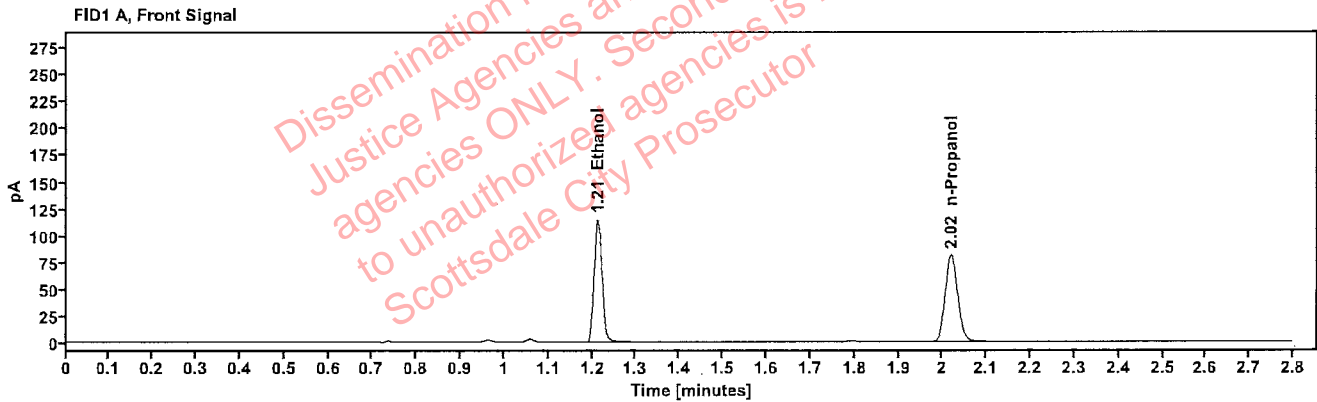


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2223	1.212	146.004
n-Propanol	-----	2.020	161.388

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	160.882
n-Propanol	2.440	176.965

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321745-2	Vial:	36
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:29:48 PM
Data file:	C:\Chem32\1\Data\31Oct23\36.D	Analyst:	Abbott

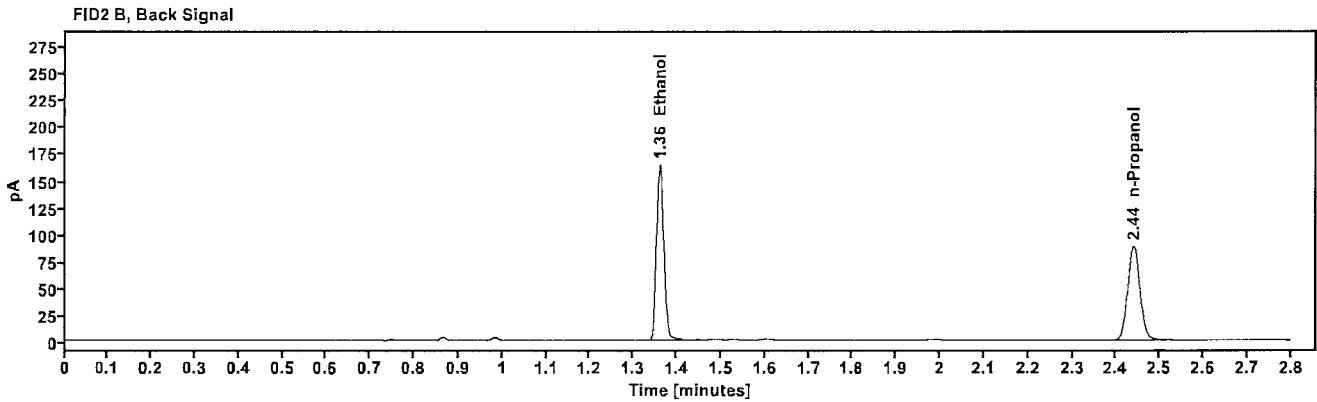
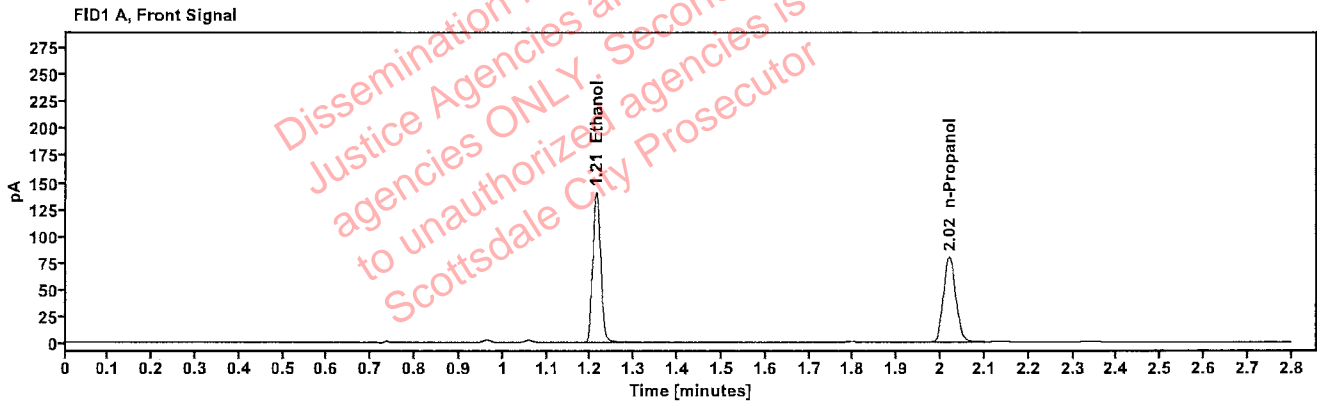


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2774	1.212	178.575
n-Propanol	---	2.020	158.220

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.360	197.982
n-Propanol	2.440	173.824

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321745-2	Vial:	37
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:34:04 PM
Data file:	C:\Chem32\1\Data\31Oct23\37.D	Analyst:	Abbott

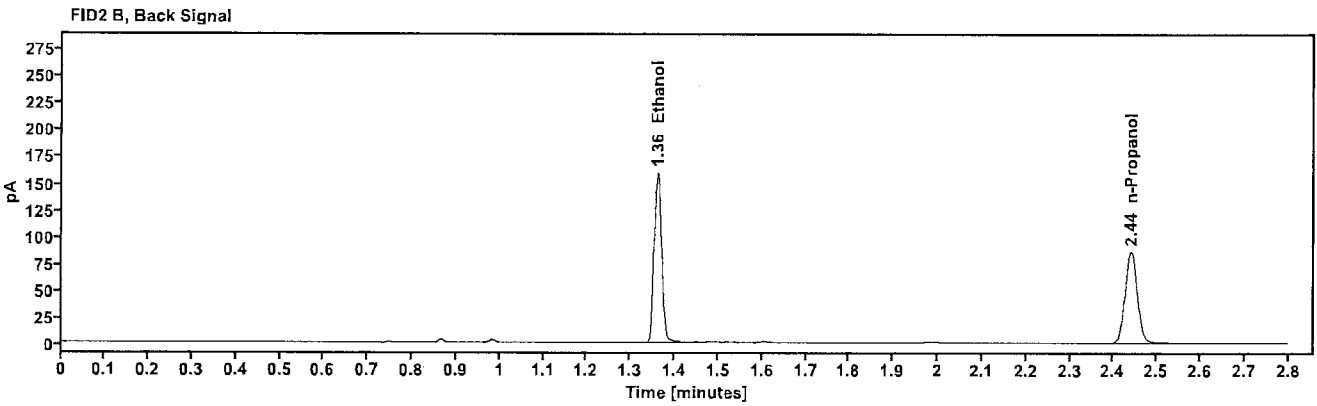
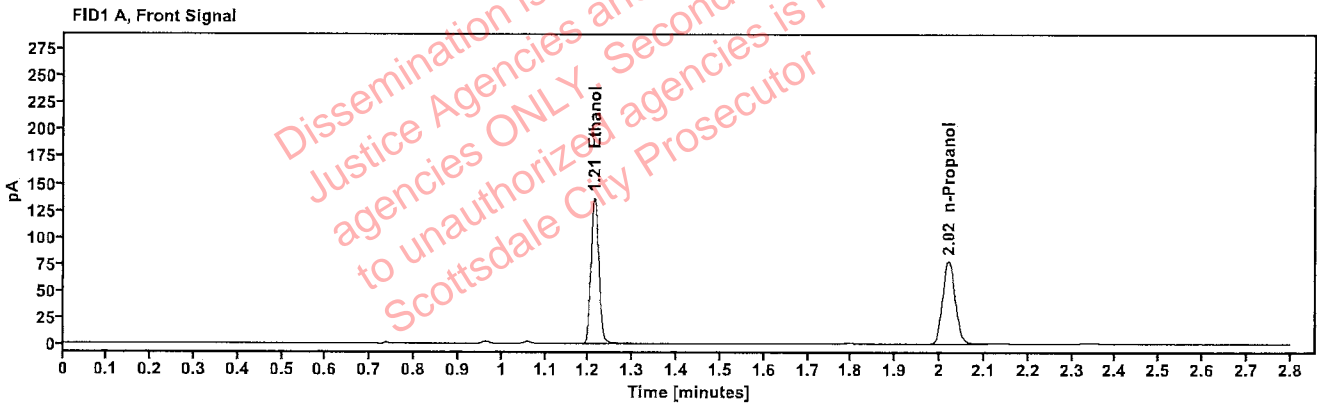


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2769	1.212	173.878
n-Propanol	-----	2.020	154.338

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.360	192.755
n-Propanol	2.440	169.518

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321859-1	Vial:	38
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:38:04 PM
Data file:	C:\Chem32\1\Data\31Oct23\38.D	Analyst:	Abbott

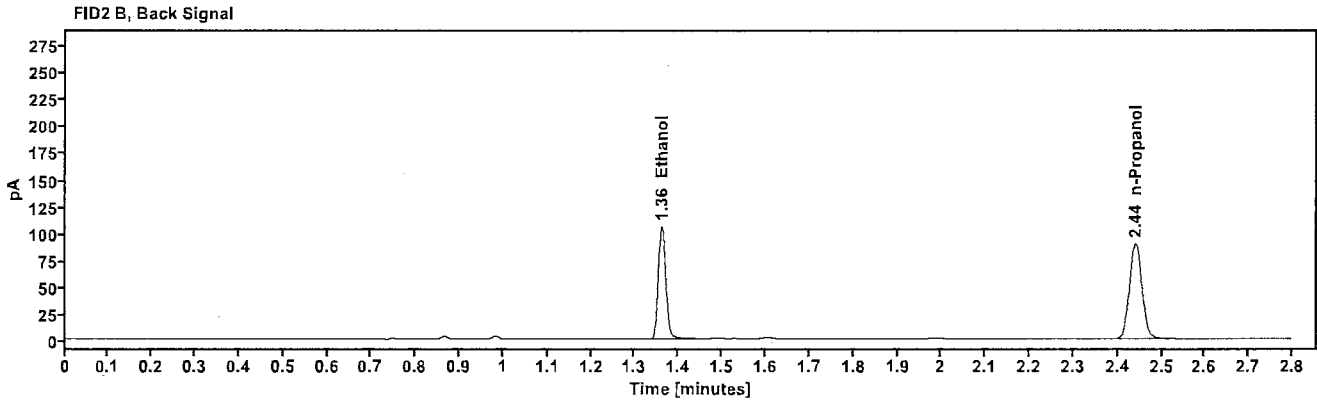
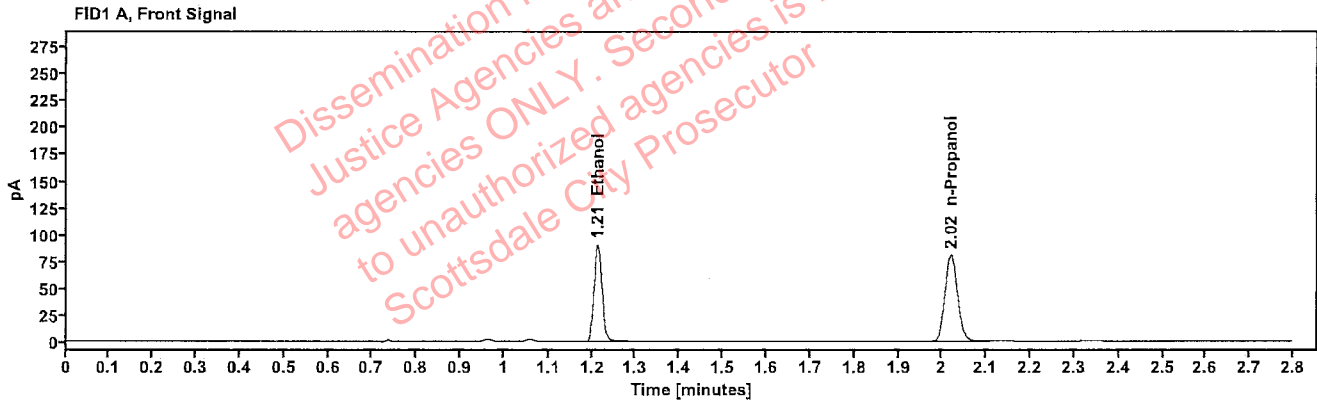


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1753	1.213	115.177
n-Propanol	-----	2.020	161.310

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	127.093
n-Propanol	2.440	176.095

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321859-1	Vial:	39
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:42:04 PM
Data file:	C:\Chem32\1\Data\31Oct23\39.D	Analyst:	Abbott

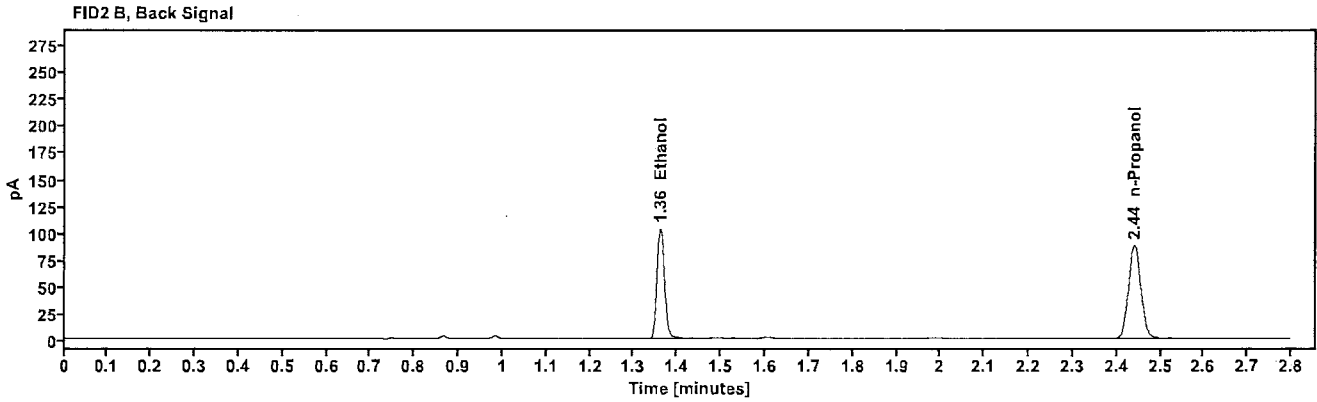
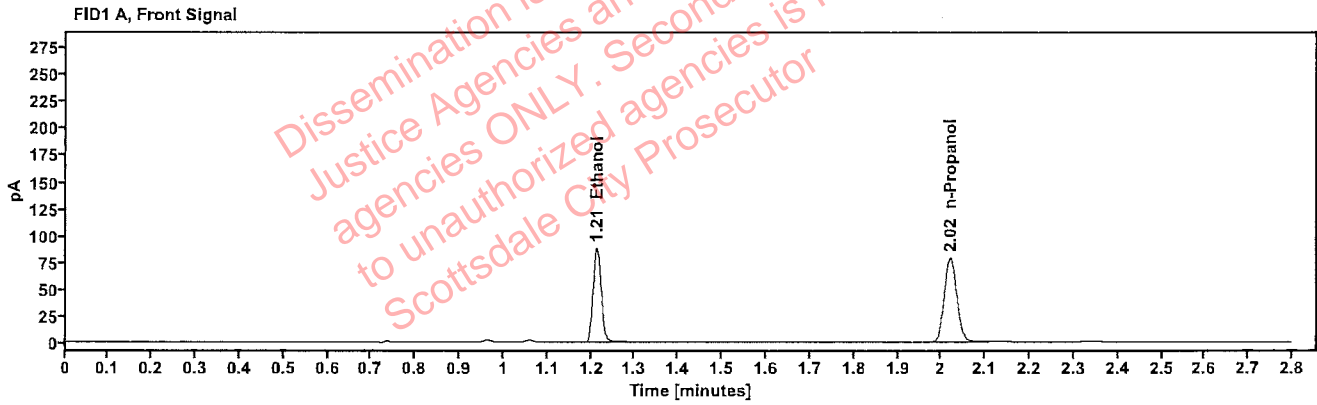


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.1748	1.213	112.531
n-Propanol	-----	2.020	158.095

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	124.209
n-Propanol	2.440	172.571

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321905-2	Vial:	40
Description:		LIMS ID:	
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:46:03 PM
Data file:	C:\Chem32\1\Data\31Oct23\40.D	Analyst:	Abbott

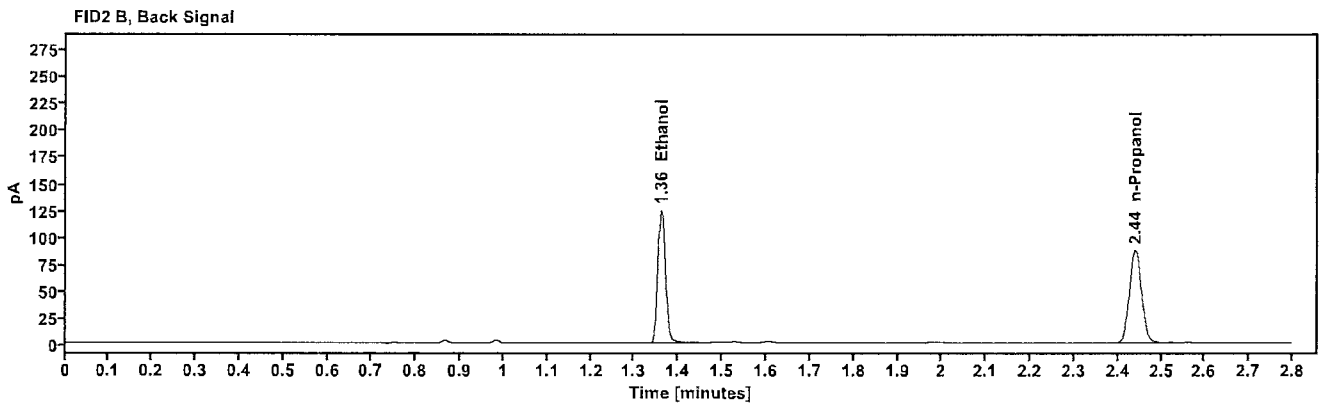
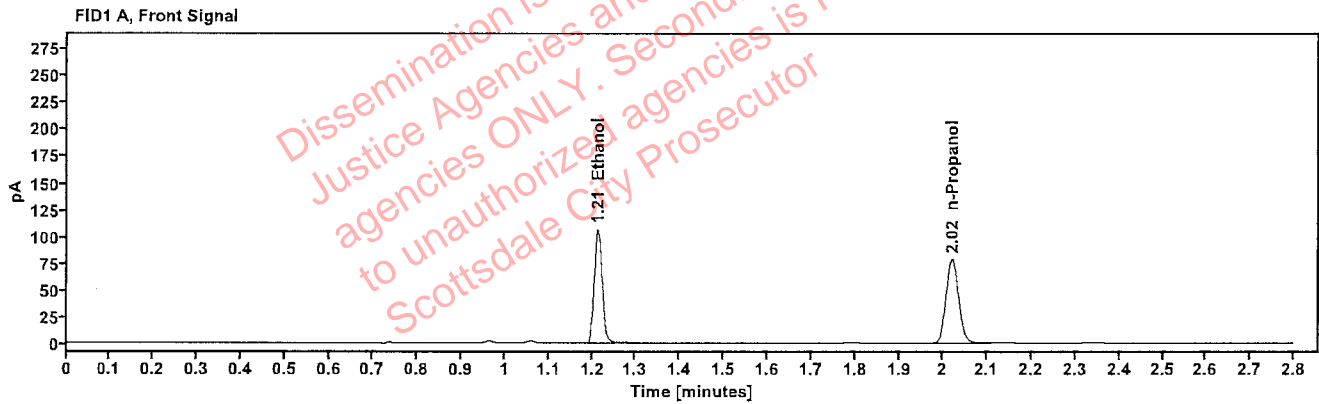


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2121	1.213	135.568
n-Propanol	-----	2.020	157.016

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	149.471
n-Propanol	2.440	171.118

Scottsdale Police Department Crime Lab Volatiles Analysis

Sample:	2321905-2	Vial:	41
Description:	[REDACTED]	LIMS ID:	[REDACTED]
Method:	ethanol quant.M	Sequence:	31Oct23
Instrument:	US14173023 CN14160045	Injection date:	10/31/2023 6:50:04 PM
Data file:	C:\Chem32\1\Data\31Oct23\41.D	Analyst:	Abbott

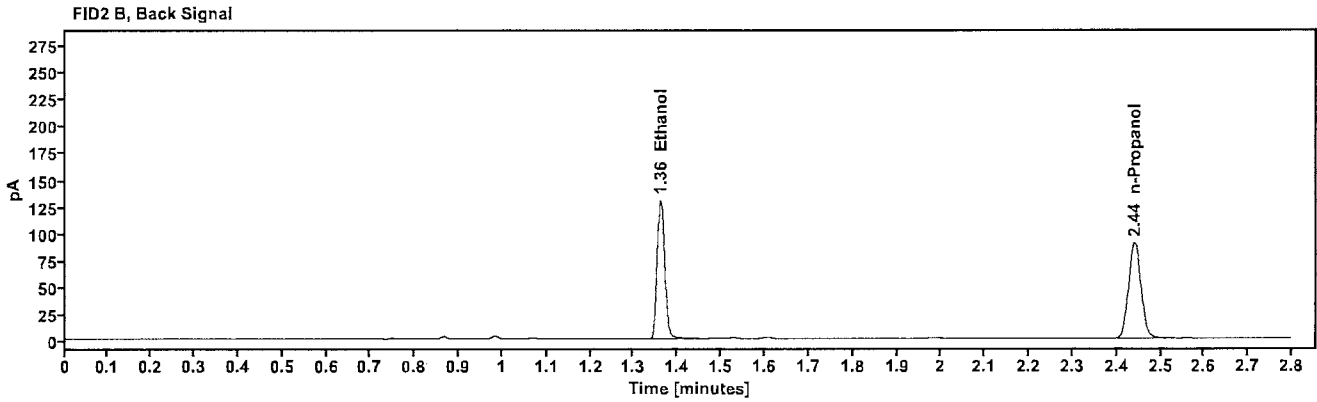
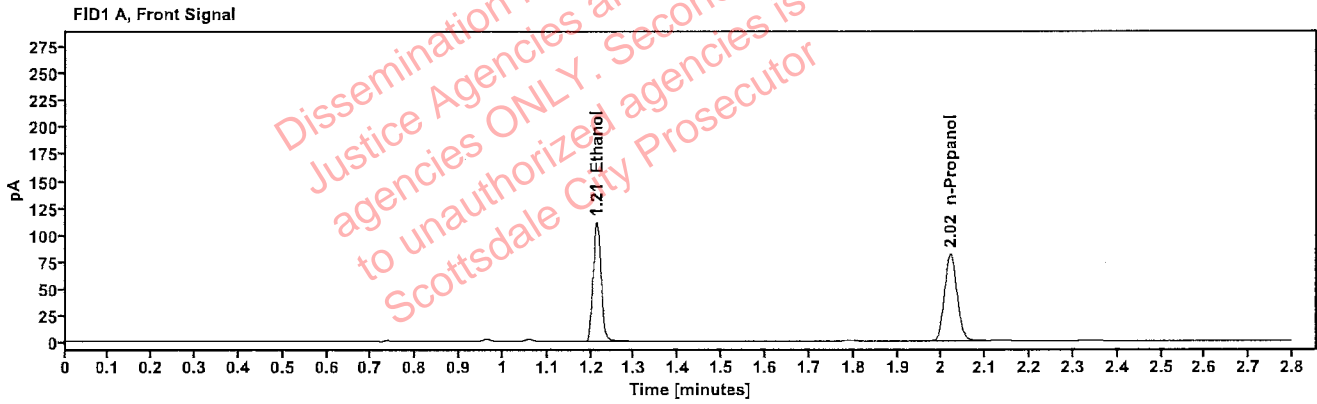


Table 1: FID 1 A (column DB-ALC1)

Compound	Amount (g/100mL)	Time (min)	Peak Area
>Ethanol	0.2146	1.213	141.865
n-Propanol	---	2.020	162.411

Table 2: FID 2 B (column DB-ALC2)

Compound	Time (min)	Peak Area
Ethanol	1.361	156.476
n-Propanol	2.440	177.284